

Periodic inspection, Slings

Guidelines for the periodic inspection of Human Care slings (non rigid body support units) according to ISO 10535:2021, Appendix Annex B.

To ensure safety and good function, the sling must be checked before each use. In addition, a documented inspection should be carried out at least every 6 months.

More frequent inspections can be required if the sling is used or cleaned more frequently than normal.

Periodic inspections should be carried out by a person who is suitable and well qualified and has good knowledge of the construction, use and maintenance of slings/person lifts.

Information	
Sling / Article number / Size	
Manufacturing date / Serial number	
Name of the user	

Inspection/control of sling	Answer / comment:
Is the label intact and readable?	
Is the sling's fabric intact? - No tears or holes? - Abnormal wear?	
Are the sling's seams, edges and loops all intact?	
Is there a user manual available?	
Is there a fitting protocol available?	

Any further comments:

Approval/disapproval of sling	
Date of inspection:	
Inspection carried out by:	
Signature of inspector:	 <hr/>



Destroy and throw out the sling if it is not approved after the periodic inspection!