HUMAN CARE
Your Life. Your Way.

Dolphin II Commode, Shower chair /
Dusch-, toalettstol / Dusch-, WC-Stuhl /
Douche, Toiletstoel 90561

User manual / Bruksanvisning / Gebrauchsanweisung /
Gebruikershandleiding

Content

ENGLISH

User manual	3
Intended use	4
Included as standard	6
Accessories	6
Safety warnings	7
General maintenance and use	8
Safety instructions	9
Parts	11
Technical specifications	12
Product label	14
Product symbols	14
Description of parts	15
Use	22
Cleaning and maintenance instructions	24
Troubleshooting	27
Recycling	28

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User manual

Important!

Always read the user manual before using your product and keep this manual for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the product itself.

INFORMATION symbol provides recommendations and information for the correct and easy use.

SAFE WORKING LOAD symbol indicates the maximum client weight for the product. Do not exceed this weight under any circumstances.

CE MARKING symbol indicates that the product is a medical device according to the definition of a medical device in EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product complies with the requirements of EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

RECYCLE according to local regulations.

INSTRUCTIONS do not use this product until you have read and understood the instructions.

Product modification is not permitted. Always check before use that the product is not damaged and shows no signs of wear and tear.

At Human Care, we continue to develop and improve our products in order to offer our customers the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the latest version of the user manual. It is available on our website, www.humancaregroup.com.

Please contact your local dealer if you have any questions about the product and its use.

Dolphin II

Intended use

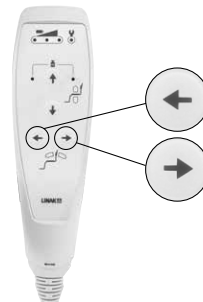
The Dolphin II commode/shower chair is a mobile, electric height adjustable tilting commode/shower chair. This chair is intended for carrying out daily personal hygiene activities such as washing, showering, toileting, changing and dressing in the most comfortable manner. While the client is sitting in a relaxed, possibly tilted position, the optimal height adjustment function prevents the caregiver from having to stoop unnecessarily. The Dolphin II is a mobile unit and can be used to transport a client over a short distance indoors. Because of its small turning radius, the Dolphin II is also suitable for small spaces.

The seat can be raised and lowered using the hand control, in order to achieve the optimum height for working and in relation to toilet visits. The motor of the chair raises, lowers or tilts the client. This means that the Dolphin II has a large range: low for easy entry and high to reach the correct working height (hip height) for the caregiver, so that even the personal hygiene care of the lower body and lower extremities can be carried out at hip height.

The seat and backrest tilt as one, keeping the ergonomic hip angle of 90° and allowing the client to remain stable while seated. The electrically operated tilt moves seamlessly from -5° to +26°. This makes it easy to find a safe and comfortable position using only one hand while the client is showering and without any effort either by the client or the caregiver. This chair also has a forward -5° tilt. This forward tilt means it is easier for the client to stand up from or sit down into the chair.

The Dolphin II is easy to operate using the four functions on the hand control.

The Dolphin II has a headrest that can be adjusted in terms of height, depth and angle. The bowl-shaped headrest means that the client can rest their head securely and comfortably on the headrest.

The height of the footplate can be adjusted to provide correct foot support for the client. The footplate has an anti-slip coating to make it easier to keep the feet on the footplate. The footplate can be folded up so that it does not get in the way of a transfer in/out of the chair.

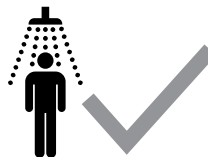
The chair can be positioned over most toilets (especially a wall hung toilet, so that the cistern is not in the way) and can also be used with a toilet bucket.

Due to its compact size, the Dolphin II is ideal for use in small spaces, both at home – narrow doorways and toilets/bathrooms – as well as in healthcare institutions where other necessary medical devices can often clutter an area. The chair is easy to maneuver thanks to the large, high-quality twin castors. The four castors make it possible to move the chair over short distances indoors, e.g. from bed to bathroom and vice versa. Double brakes on the castors ensure a stable position during personal hygiene activities and during transfers into/out of the chair.

This, and many other factors, make Dolphin II, a durable quality product. This commode/shower chair is used in private homes, care institutions, nursing homes, hospitals, but also in swimming pools and multi-sensory stimulation rooms.

 This product may not be used under water, such as in a bath or swimming pool.

The chair can be adjusted to the height of the user by adjusting the neck/headrest and footplate.

Dolphin II

Included as standard

- Dolphin II 1x
- User manual 1x
- Charging cable 1x
- Battery 1x

Accessories

Photo	Description	Article number
Bucket 	Toilet bucket, including lid	04-M0312-0016
Toilet seat with cover 	Toilet seat with cover for the keyhole-shaped opening for extra comfort	04-M0307-0009
Battery 	Extra Linak battery to ensure the chair is always ready for use	04-N0101-0012
Wall mount 	Wall mount for external charging	04-N0901-0043
Safety belt	Safety belt	90562

Safety warnings

Only trained professionals can make the decision whether or not this product is appropriate to use. This product can only be used under the supervision of a caregiver. Never leave the client unattended.

If you do not understand any of the warnings, safety precautions or instructions, please contact a care provider, distributor or technical staff member (if applicable) before attempting to use this product. Using this product without having the proper knowledge may result in injury or damage.

Disconnect the product and remove the battery during maintenance work.

Product modifications are not permitted. Maintenance and servicing may only be carried out by qualified personnel.

Only use the parts that are supplied with the product. Only use spare parts supplied by Human Care.

The maximum safe working load is 180 kg/396 lbs.

Do not touch parts of the products that may have an electrical current (leakage) at the same time as the patient.

Dolphin II

General maintenance and use

Check the Dolphin II before each use to make sure it is not damaged in any way. Do not use the Dolphin II if there are signs of wear and tear or damage on any part of the chair.

This product may only be used by qualified personnel.

The Dolphin II may only be used on an even, clean and hard-floored surfaces that are free of obstacles. The product should never be used on a sloping floor or ramp.

In certain circumstances, more than one caregiver may be necessary to assist a particular client.

The client's abilities should be assessed before using this product. Only use this product for its intended use.

The Dolphin II is designed to support only one client at a time. Never use the product to lift a client and an item of furniture at the same time.

Only use other Human Care products with the Dolphin II.

This product is for indoor use only. It is not suitable for use in areas with high humidity, extreme temperature or high pressure environments.

A specialist must approve use of the product for a specific client and this must be recorded in writing.

Before using the product with a client, it is strongly recommended to practice using the product with colleague-caregivers, until the caregiver feels comfortable using the product.

The correct type, size, material and design of accessories must be carefully selected by an expert, taking into account the safety and needs of the client.

Always charge batteries in a well-ventilated room. Check the charger regularly.

Safety instructions

- Please read the instruction manual for both the commode/shower chair and the accessories before use. Lifting and transporting a person always involves a certain risk.
- It is important to fully understand the content of the manuals and to use the product according to the instructions in the manuals.
- The product may only be used by trained personnel.
- In case of doubt or if you have any questions, please contact Human Care or your distributor.

Warning!

Certain conditions and surroundings may limit the use of the devices, e.g. thresholds, uneven surfaces, obstacles and deep-pile carpet. These conditions and surroundings may prevent the castors of the commode/shower chair from working properly, may cause the commode/shower chair to become unstable and/or may make it harder for the caregiver to use. If you are not sure whether your care environment meets the requirements for proper use of the product, please contact Human Care for advice.

Make sure that fingers, hair, toes, etc. cannot get caught between the moving parts as these can become trapped when adjusting the position of the chair.

CAUTION!

This user manual contains important information for the user of this product. All users of the product must read and understand the content of the user manual and work accordingly. Always keep the user manual in a place where it is available to those persons using the product.

CAUTION!

The assembled seating unit may only be raised by the lift actuator. Manually lifting the seating unit could damage the lift actuator.

Do not use the product in an environment where other radio frequency emitting devices are used. Like any other electronic device, the product can use and emit radio frequency energy. If the product is not used in accordance with the instructions in the user manual, it may cause interference with nearby devices. The manufacturer cannot guarantee that interference will not occur in a particular location. In order to check if the product is the source of any interference, change its location or disconnect the battery. If you still notice or suspect a fault, please try the following:

- Move the lift to another location;
- Increase the distance between devices;
- Contact the service-desk.

Dolphin II

General safety precautions

Never take the product down a flight of stairs, either with or without a client.

Do not try to use the product on a slope of more than 5 degrees (approximately 1:12) when transporting a client.

Do not try to go up a slope without a second carer.

Before moving the product, always check that it is disconnected from the charger, as this could damage the electrical connection.

Always make sure you follow these instructions before using the lift:

Ensure the batteries are fully charged;

Always read the instruction manuals for the product and the accessories;

Personnel working with the product are aware of the correct operation and use of the product;

Always carry out a daily check before using the product;

The caregiver is familiar with the controls and safety devices before attempting to care for a client;

Before using the product, you must ensure that:

You know what you need to do before starting;

The accessories are not damaged;

The accessories are correctly attached to the product;

The correct accessories are being used in terms of type, size, material and design to meet the client's needs;

Never leave a client unattended while in the product!

Parts

This product can be used in damp areas, such as a bathroom.

However, it is not suitable for bathing.

The product has an IP rating of IPx5 against water, which means that it can resist a sustained, low-pressure water jet spray (waterproof).

1. Hand control

The push buttons on the hand control allow for the height and tilt of the chair to be adjusted, also shows the battery and service indicators. The hand control can be stored by hanging it on the push handle.

2. Head rest

PUR (polyurethane) headrest that is height, depth and angle-adjustable in order to provide optimal support of the neck and head.

3. Seat

Removable PUR seat with in integrated rail for a toilet bucket.

4. Foot plate

Safe support of the client's feet, can be folded up and height-adjustable.

5. Castors, front (on both sides)

Twin castors, with brakes.

6. Chassis

The chassis support the seat and has 4 castors. There are 2 actuators for adjusting the height of the seat.

7. Castors, rear (on both sides)

Twin castors, with brakes.

8. Actuator (on both sides)

Electric actuator for height-adjustment of the seat.

9. Control box with emergency stop and battery

Motor control and power supply, covered for protection against water.





10. Upper frame

This is the seating for the client, equipped with a removable seat, backrest, folding armrests, footplate and headrest. This is also where the control box, battery and actuator for the tilting function are located.

Dolphin II

Technical specifications

Description	
Article number	90561
Length of the frame	92 cm / 36.2"
Width of the frame	67 cm / 26.3"
Height	102 cm / 40.1"
Height of the push handle	102-131 cm / 40.1-51.5"
Size of the seat	42x43 cm / 16.5x 16.9"
Seat height	48-100 cm / 18.8-39.3"
Tilting angle of the seat	-5° to +26°
Diameter of the castors	12.5 cm / 4.9"
Product weight	60 kg / 132lbs
Maximum user weight	180 kg / 396lbs
Turning radius	110 cm / 43.3"
Lifting speed up (0/maximum load)	42/49 sec.
Lifting speed down (0/maximum load)	47/41 sec.
Control box, input voltage	input voltage 100-240V ~, frequency 50/60 Hz, output voltage 24V, power consumption max. 400 mA
Battery capacity	2.9 Ah
Electric shock protection class	II, adapted part type B
IP rating class	IPX5
Frame material	Powder-coated steel
Operation	Continuous, short-term load (10%) max. 2 min. in operation (on), min. 18 minute break (off)
IP rating control box	IPX5
IP rating class battery	IPX5
IP rating main lift actuator	IPX6
IP rating hand control	IPX6

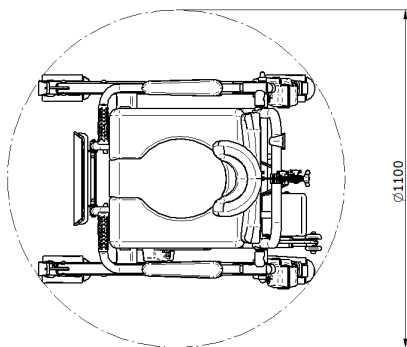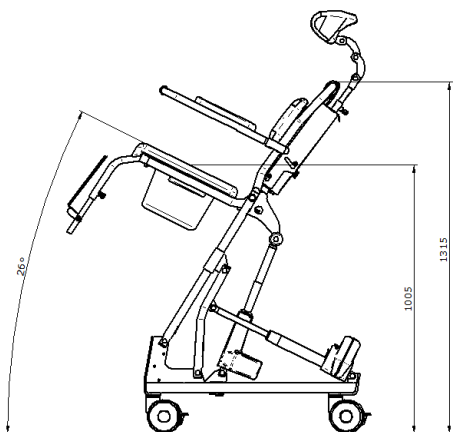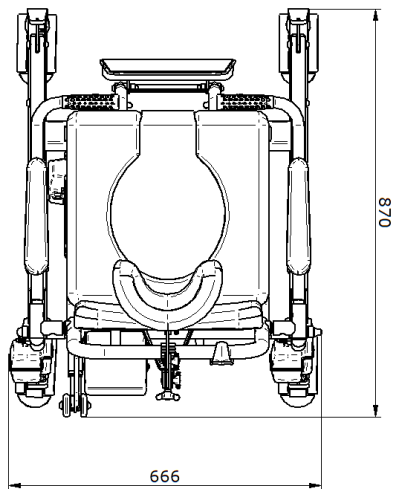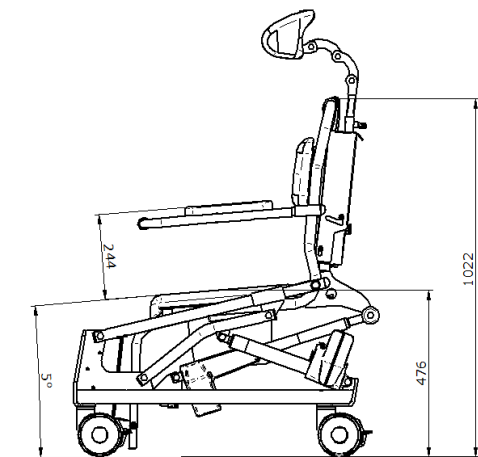Dolphin II

Product label

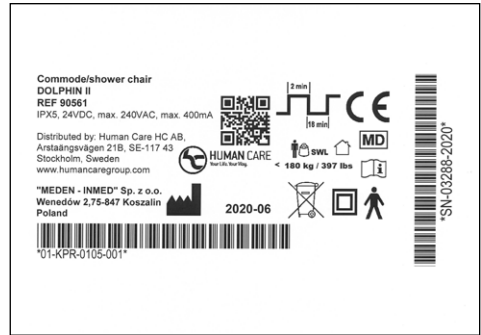
The product label contains information about the product and is located on the right-hand side, close to the mast.

Human Care is an ISO 13485:2016 Swedish certified medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product has been tested in accordance with ISO 17966:2016.

The product has a CE marking in accordance with the Medical Device Directive 93/42/EEC and/or Medical Device Regulation (EU 2017/745). The CE mark label is at the product.

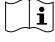
This product has been tested by an accredited test institution and meets the following necessary medical requirements: EN 17966:2016, EN 60601-1 (electrical safety) and EN 60601-1-2 (EMC). This product carries the CE marking.

Product symbols

Symbols on the product and the product label:

	Product operating cycle
	CE marking
	MD Medical Device
	Product manufacturer
	Maximum user weight

	For indoor use
	User manual: Please read the manual to familiarize yourself with its contents before using the product
	This product contains electrical and electronic components and must be recycled according to local guidelines.
	Electric shock protection class II
	Product type B: approved for physical contact with clients (IEC 60601-1)

Description of parts

Hand control

The Dolphin commode/shower chair has a hand control with the following functions:

1. Battery charge

The battery charge indicator has three status indications. The battery charge indicator is yellow or green until switched off (2 minutes after use).

Battery status

For a full description, see page 18.

2. Service indicator

The service indicator (yellow indicator) flashes when it is time to carry out a service. The default setting is every 12 months / 8000 lifting cycles (as recommended by the EN17966:2016 standard), whichever comes first. When it is time for a service, the service indicator stays on for 2 minutes after use. It is then switched off to save battery life. The system is still functional when the service indicator is on.

3. Overload indicator

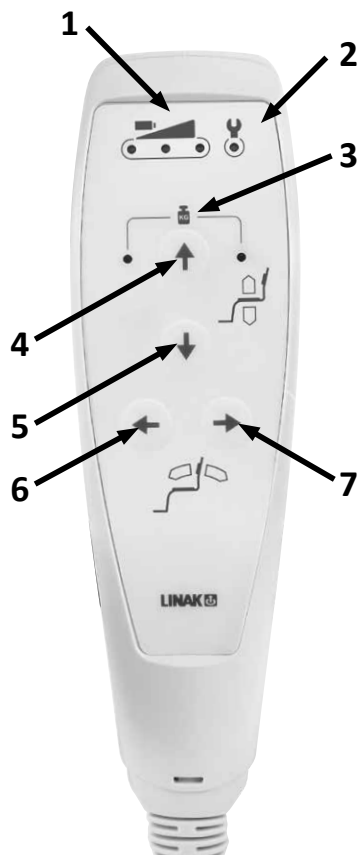
When there is an overload, according to the predefined power interruption, the two LED lights will flash for 10 seconds.

4. Up arrow

Moving the seat up.

5. Down arrow

Moving the seat down.

6. Left arrow

(from the reader's perspective) Tilting the seat forward.

7. Right arrow

(from the reader's perspective) Tilting the seat backward.

Dolphin II

Emergency stop

In the event of an emergency, immediately press the emergency stop button.

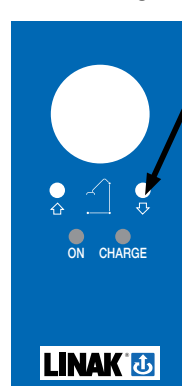
Turn the button clockwise to unlock the emergency stop.

Electric emergency lowering function


If the button on the hand control for lowering the lift arm does not work, you can use the electric emergency lowering function. This can be activated by using a pointed object (e.g. a pen) to press the small button on the control box. You will hear an audible click. Keep pressing the button until the required height has been reached.

Electrical emergency lowering

Mechanical emergency lowering function

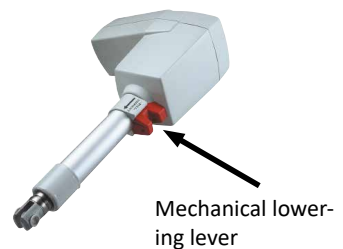If the emergency lowering function via the control box does not work, use the mechanical lowering function. This will safely lower the client in the event of a fault in the electrical system. Lowering can be done manually by using the lever located on the lower part of the main cylinder. To operate, grasp the red part and pull it up. Allow the lift arm to lower until the user can be safely removed from the lift.

Caution!

After using the mechanical or electrical emergency lowering function, you must immediately contact your distributor to schedule a visit by a service technician. The product may only be used again after an authorized employee has carried out an inspection of the product.

Battery

Installing and removing the battery

Caution!

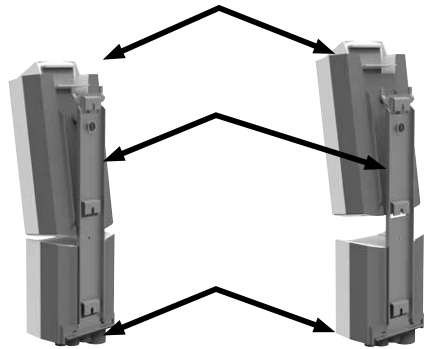
Do not use the lift for at least one hour before replacing the battery.

Never expose the battery to extreme temperatures.

Never leave the battery charger in damp/humid areas.


Installing the battery:

- Align the bottom of the battery with the top of the controller.
- Push the top of the battery forward until you hear a click. The battery is now in place.
- Gently push the top of the battery forward to make sure it is securely attached to the holder.

Removing the battery

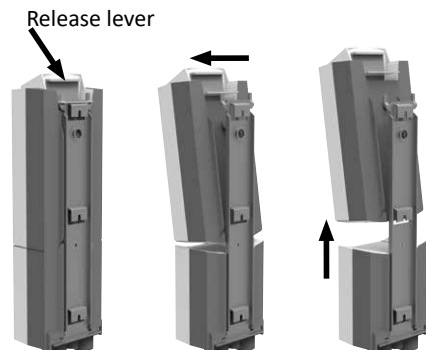- Press the red emergency stop button.
- Hold the battery by the handle and press the release lever.
- Remove the battery by lifting it off the holder.

Dolphin II

Battery charge display and charging

Caution!

- The product cannot be used while charging.
- During charging, the functions are locked.
- Do not use the lift for one hour after charging. This will prolong the life of the battery.
- The battery needs to be charged continuously for at least 24 hour, in the following situations:
 - a) Before using the lift for the first time.
 - b) Before being placed in storage (up to 3 months) without a power supply connection.
 - c) Immediately after a storage period.

Battery charge indicator

The battery charge indicator has three status indications. The battery indicators are either yellow or green until switched off (2 minutes after use).

Battery status 1:

The battery is full (100-50%) – no recharging required: the third indicator is green.

Battery status 2:

The battery is okay (50-25%) – recharging required: the second indicator is yellow.

Battery status 3:

The battery is low (less than 25%) – recharging required: the first indicator is yellow and a buzzer sounds whenever a button is pressed.

Battery status 4:

The battery requires recharging. Some functions of the lift are lost and it is only possible to lower the lift arm. When in continued use, the indicator flashes and an audio signal is emitted.

The battery charge is indicated on the hand control.

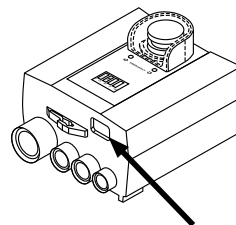
Battery level indicators on the hand control

Charging the battery

If you need to charge the battery, you need to:

- Press the emergency stop button
- Connect the black charging cable that is supplied to the control box
- Then insert the plug into a wall socket that is located in a dry room.

Connection for charging cable

When the rechargeable battery is connected, the LED display on the control box will come on.

The green LED indicates that the battery charger is connected to the mains. The yellow LED indicates the charging process. It takes approximately 5 hours to fully charge the battery.

Charging the battery with an external charger (optional).

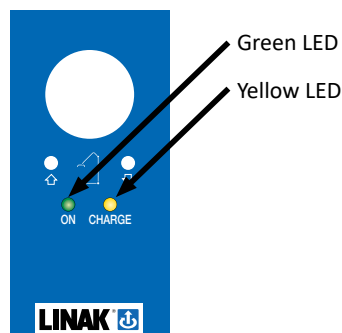
You can always purchase an external charger, for example to have a spare battery to hand. To charge the battery, it needs to be inserted in the external charging station. Ensure that the battery is properly inserted into the charging station. A light will come on if it is attached correctly.

If the battery emits a short beep, it needs to be charged. If a green light comes on when the battery is inserted, the battery is full. If the orange light comes on, the battery needs to be charged. Charging will stop automatically when the battery is full. This means you cannot overcharge the battery. It is best to charge the battery every night.

The external charger can be easily mounted by means of the wall mounting plate. It can be mounted on the wall near a 230 V socket.


We always recommend you purchase a spare battery, so that you always have access to a full battery. If one battery is empty, you can continue to work using the second battery.

Location of LED on control box

Green LED

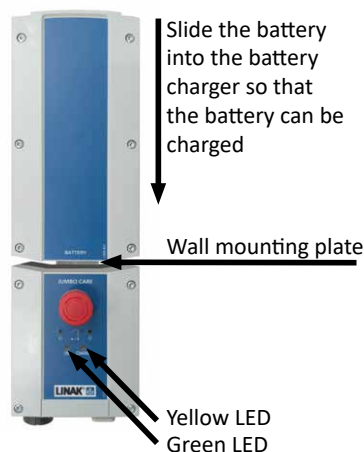Yellow LED

Slide the battery into the battery charger so that the battery can be charged

Wall mounting plate

Yellow LED
Green LED

Charging the battery must not take place in a wet/humid environment (shower area).


Dolphin II

Headrest

The Dolphin II has an adjustable PUR headrest to provide reassuring support and stability for the client's neck and head. When the client is seated in a backwards tilted position, the headrest is necessary.

The picture shows that the headrest has four attachment points which are attached by a star knob. To adjust the height, unscrew the knob (1) and simply pull it out. Adjust to the required height, e.g. at the level of the client's ears, and then tighten the knob.

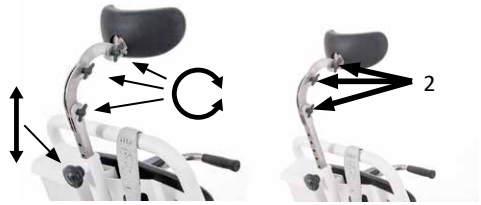To adjust the depth and angle of the headrest, unscrew the relevant knob (2), then tighten the knob once the headrest is in the preferred position.

Toilet seat with transport cover

The Dolphin II has a removable PUR seat, which makes it easier to clean the surface and frame. To place the seat, slide the seat at an angle on the frame and push until it clicks into place.

To remove the seat, pull up the front of the seat and slide it in the direction of the footplate.

The transport cover is placed on the toilet seat to cover the seat opening. This is to improve the client's comfort.

Armrests

The Dolphin II has two armrests with PUR elements, which provide comfortable support to the lower arm. The armrests continue around the front so that the client has a secure support, which is useful when it is necessary for the client to lean forward to receive personal care. This design also makes the client feel safe. The armrests are there when needed, but can be easily folded up so that they do not get in the way of a transfer in/out of the chair.

Footplate

Dolphin II has a footplate to support the client's feet, to ensure an ergonomic and relaxed posture. The anti-slip coating makes it easier to keep your feet on the plate.

Adjusting the footplate

The footplate of the Dolphin II can be adjusted in height by means of two star knobs (1), which are located at the rear on the upper frame.

The footplate itself can also be folded up by pushing the front side upwards so that it does not get in the way during a transfer (2).

When the chair is fully lowered, the footplate must be folded up.

Brakes on the castors

The Dolphin II is fitted with four twin castors with brakes, i.e. the brake locks in forward/reverse and direction of rotation. These brakes guarantee a stable position when daily activities are being performed.

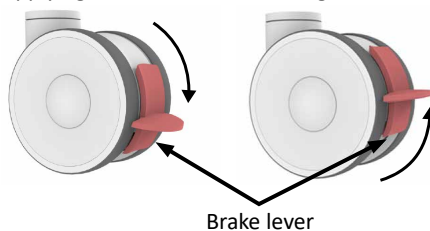
The brakes on the castors are easy to operate by pressing the red brake lever with your foot. To release the brakes on the Dolphin II, move the red brake lever upwards.

 **Warning:** Do not stand on the footplate!

Four castors with brakes

Applying the brake Releasing the brake

 **Warning:** The brakes must be checked before each use.

Dolphin II

Use

Installation and use of the Dolphin II

Please make sure you have gone through the following checklist before using the product:

The Dolphin II is designed to be used on an even, solid and level floors and is intended for indoor use.

Make sure that your route is free of obstacles; furniture and other items must be cleared from the transport route. Keep the area free of obstacles.

Be careful with each transfer and if necessary, consult the specific instructions for the client.

The condition of the client must be monitored during the transfer and other activities in the product.

This product may only be used by trained qualified personnel.

If possible, the operation of the product should be explained to the client.

Moving the Dolphin II

When moving with the chair, it is recommended to keep the chair at its lowest possible height and for the seat to be in a neutral position of 0-5°. The castors are not braked.

The push handle at the back must be used for moving and maneuvering the Dolphin II. Think about your posture:

- Walk behind the product when pushing it straight ahead.
- Start moving the product by using your own body weight as much as possible.
- Keep your back as straight as possible.

Operating the Dolphin II

- Pull out the emergency stop on the control box before use.
- The Dolphin II is now ready for use.
- Press the appropriate button on the hand control.

Emergency stop

- Comment: The battery switches itself off after three to four minutes of inactivity. This is to avoid unnecessary drain on the battery.

Use

- Move the chair to the client.
- Place the seat in the correct position in relation to the client for a safe transfer, with or without a device.
- Adjust the seat of the chair to the correct height and angle for the transfer.
- The client sits down into the chair.
- Adjust the chair for the client: headrest in correct position and footplate at correct height
- You can now start the required activities, such as:

1. Washing, drying, dressing:

It is recommended that the chair is adjusted to the most comfortable working height for the caregiver while carrying out the personal hygiene activities.

Engage the brakes of the castors. The seat can be tilted backwards to provide a comfortable position for the client.

2. Toileting:

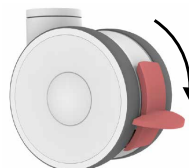
Move the chair over the toilet. Make sure the distance between the toilet and the seat is as short as possible to prevent splashing. Engage the brakes of the castors.

The footplate can be raised to make toileting more comfortable.

- When getting out of the chair, the brakes of the castors must be engaged to ensure the chair is stable. The footplate is folded up. The seat can be tilted forwards, $-5-0^{\circ}$, making it easier to get out of the chair. The transfer can be carried out with an aid. If preferred, the arm rests can be folded up.

Applying the brake

Cleaning and maintenance instructions

Cleaning and maintenance are important requirements to ensure that the product works properly over a long period of time.

Periodic maintenance

Maintenance and repairs of the Dolphin II must be carried out by a service technician approved by the manufacturer. It is recommended to arrange for a maintenance contract with a certified company to ensure that the Dolphin II is always properly maintained.

The Dolphin II must have a maintenance check every year.

All moving parts must be lubricated once every six months or when they start making a loud noise when used.

These moving parts include the castors, actuators and bearing bushes. As a lubricant, we recommend using commercially available penetrating and lubricating products (e.g. Wurth HHS 2000). Any spillage of such lubricants must be removed immediately with a dry cloth.

Any unwanted tolerance must be remedied. Any tolerance in joints that cannot be remedied must be reported to the manufacturer's service department. The product can no longer be used until the cause of this slack has been removed.

CAUTION!

Human Care cannot be held responsible for damage caused by willful damage, overdue maintenance and incorrect use, or any resulting consequential damage.

Daily cleaning and maintenance

Charge the batteries when the Dolphin II is not in use.

Before charging the battery, check that the charger's power cable and plug are in good condition.

If you detect any damage to the charger, you must immediately disconnect it.

We recommend checking the product regularly for loose parts and to fix these.

Regularly clean the surface of the castors with a damp cloth and suitable cleaning agent.

Check the hinged parts of the seat and the swivel castors.

After completing a transfer, the Dolphin II should be wiped with a damp cloth soaked in a detergent.

Completely dry the Dolphin II after use in the shower.

CAUTION!

The Dolphin II must be cleaned and disinfected every time a different client uses the chair before it can be used again.

**CAUTION!**

Before cleaning the lift, check that:

- All plugs are correctly connected.
- The battery is mounted on top of the control box.
- None of the electrical components show external damage. Failure to do this may result in water or cleaning agents to get into the chair and cause interference with the unit or damage to electrical components.
- Electrical components cannot be washed with water jets or high-pressure cleaners, etc. They can only be cleaned with a damp cloth.
- Stop using the product if you suspect that water or liquid has entered the electrical components and report the incident immediately to the service center.
- Failure to comply with the above rules may result in serious damage to the equipment and could have other unforeseen consequences.

Instructions for cleaning

Observe the instructions of the detergent's manufacture when cleaning.

If necessary, use suitable protective clothing such as aprons, goggles and gloves.

Any large stains must be removed immediately.

The frame can be cleaned with a damp cloth.

Use a mild and environmentally friendly household cleaner for stubborn stains.

Do not use abrasives and corrosive substances, such as pastes, waxes, sprays, strong detergents, solvents and detergents containing solvents, alcohol and leather cleaners. These types of cleaning agents can also cause stiffness and cracks in the material and alter the surface shine, this is not covered by the warranty!

Please make sure that all contact surfaces of the Dolphin II are thoroughly cleaned.

Routine cleaning of the product will suffice if it is used by the same client.

Instructions for disinfecting

The product will need to be disinfected if there are visible signs of material or possibly infected material (blood, stool, pus) or in situations requiring a high level of high infection control.

Please observe the instructions of the manufacturer of the disinfectant.

The Dolphin II can be disinfected with household disinfectants.

All surfaces can be wiped clean with a disinfectant.

The castors only need to be disinfected if there are visible traces of infected or potentially contaminated material.

Rinse off the disinfectant and leave to air dry.

Subsequently, the Dolphin II should be inspected for cleanliness and damage.

Dolphin II

Storage and transport

The product must be kept in a cool and dry room. The climate conditions of the room must be within the following parameters:

- Temperature 10-40 °C (recommended 20 °C or lower);
- Humidity 30 ÷ 75%;
- Air pressure: 700 ÷ 1060 hPa.

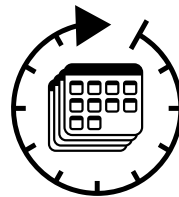
Do not use the product in environments where flammable materials may be present, e.g. in storage areas for flammable substances.

During long periods of inactivity, we recommend checking all electrical and mechanical parts once a month by carrying out a functionality test without the client. Additionally, the charging cables should be visually checked for possible damage after each mechanical load or after moving the product to a different location.

Periodic electrical safety tests

A service is required every 12 months or 8000 cycles, whichever comes first and every time after a fault/repair of the product. Services, including the electrical safety test, must be carried out by a certified person. Services should include as a minimum:

- A check on whether there is mechanical damage to the wiring;
- A check on whether there is mechanical damage to the housing of the controller, battery and actuator;
- A check on whether height adjustment and tilting adjustment buttons are working properly.

Troubleshooting

Problem	Solution
The product does not respond when the buttons on the hand control are pressed.	Check whether the charging cable is connected to the product and the mains
	Check whether the emergency stop has been pressed
	Check whether the battery is sufficiently charged
	Check whether the service LED is flashing
	Check whether the hand control cable is properly connected to the control box
	Check whether the battery is correctly fitted
	Check whether the other wires are connected
	Check whether the chair responds to the raising/lowering button on the controller
Contact the service department	
The product is not responding to the control box.	Check whether the charging cable is connected to the product and the mains
	Check whether the emergency stop has been pressed
	Check whether the battery is sufficiently charged
	Check whether the service LED is flashing
	Check whether the battery is correctly fitted
	Check whether the other wires are connected
Contact the service department	
The battery is not charging.	Check whether the charging cable is connected to the product and the mains
	Check whether the emergency stop has been pressed
	Check whether the battery is correctly fitted
	Check whether the service LED is flashing
	If you have an external charger, check whether it is suitable for the battery.
Contact the service department	
Interruption during tilting or height adjustment.	Check whether the overload indicator is flashing
	Check whether the battery is sufficiently charged
	Check whether the emergency stop has been pressed
	Check that the hand control cable is properly connected
	Check whether the service LED is flashing
	Check whether the product responds to the raising/lowering button on the control box
	Lower the product manually by pushing a pen into the small button for the lowering function
	Contact the service department
The product is making strange noises.	Contact the service department
The product cannot be moved.	Check whether the brakes on the rear castors are locked
	Contact the service department

If the error messages persist, stop using the lift immediately and contact the distributor for further instructions.

Dolphin II

Recycling

The Dolphin II is an electronic appliance covered by Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). The product label shows the WEEE symbol, which consists of a crossed-out mobile bin. This label indicates that the batteries must be collected separately at the end of the product's lifecycle. This requirement applies to the European Union. Do not dispose of these products in household waste.

Caution!

Batteries and electronics contain lead and must be recycled according to WEEE 2002/96/EC.

Metals

Electrical and electronic waste (WEEE)

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