HUMAN CARE
Your Life. Your Way.

seBastian II 90554

Service manual / Servicemanual / Servicehandbuch /
Servicehandleiding

Content

ENGLISH

Service Manual	3
Compliances and Standards	4
Useful information	4
Service manual description	5
Tools & Maintenance	5
Troubel shooting & Solving	7
Spare parts list	13

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Service Manual

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CE MARKING symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

RECYCLE according to local regulations.

READ INSTRUCTIONS and ensure you fully understand them before using this product.

Product modification is not permitted. Before use, always check that the product is not damaged or worn.

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at www.humancaregroup.com.

Contact your local distributor if you have any questions about the product and its use.

Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This sling has been tested in accordance with ISO 10535.

Useful information

a) Service manual

The service manual is created for technicians involved in servicing of seBastian II stand-up lift. Safe performance of lift service procedures requires experience, technical skills and general knowledge of safety rules.

This manual uses warnings and cautions to pay special attention to service procedures.

Warning – this information tells us about the possibility of safety risk of technician and user.

Caution – this information tells us about actions that can cause potential damage of the lift.

b) seBastian II installation and operating

For installation and operation of the lift, see the user manual.

c) Service of seBastian II stand-up lift

The service can be only made by Human Care's authorized service team or authorized units by Human Care.

Human Care shall bear no responsibility for the consequences of repairs or modifications that have been done by unauthorized personnel and for the use of non-original replacement parts dedicated by Human Care for lifts.

d) Service documentation

Service documentation and service repair document are available on a website accessible for authorized service.

e) Safety precautions

The stand-up lift design provides maximum of safety. Safety of the device are ensured by safety standards IEC60601-1.

Read these cautions carefully before start servicing!

Note - do not perform service repairs and functional testing when the client is on the device.

f) Additional information

In case of any problems with stand-up lift contact the intermediary or manufacturer's service.

Service contact details:

HUMAN CARE SWEDEN (HQ)

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117 43 Stockholm

Phone: +46 8 665 35 00

Fax: +46 8 665 35 10

Email: info.se@humancaregroup.com

Service manual description

Service manual contains information about repairing, maintaining and identification of the stand-up lift. The lift can be identified by using

the identification code which is placed on the name plate.

Tools & Maintenance

Note – if the plug from battery charger does not match a socket in your country, you can use an adapter.

Tools

Standard set of tools is required for servicing of the stand-up lift:

Allen key set

Flat screwdriver

Hammer

Ordinary set of various keys

Volt meter

seBastian II Service Manual

Device maintaining

All movable joints must be lubricated every six months or when experiencing squeaks or other noise. These nodes are:

- axle transport wheels,
- axle actuator mounting,
- jib arm bearing sleeve.

We recommend using any available penetrating lubricants.

Report all loose connections that are supposed to be fixed to the manufacturer's authorized service agent and stop using the product until the problem is resolved.

This device is equipped with electric actuator which allows to adjust the height of the jib arm. In order to ensure proper performance of the actuators check the actuators for unusual sounds every 3 months. These unusual sounds contain rattles, squeaks, creaks. Any perceived performance deficiency must be reported to Human Care technical service immediately ceasing device operation until remove their causes.

Not less frequently than every 1 years and after each failure/repair of actuator technical service must check the lift for electric safety.

The minimal range of the review should include:

- check if there were no mechanical wiring damages of the device,
- check if there were no mechanical damages of the controller and the actuators.

See checklist in user manual.

Basic periodic review

One year after production date or every 8000 cycles of using the lift, the indicator lamp light up.

If this LED is on, operator must check condition of the lift (cables connections, looseness on the frame etc.).

Press and hold "up" and "down" button for 5 sec., to turn off the light after review.

Transport wheels

Actuator mounting

Jib arm bearing sleeve

LED indicator

Troubel shooting & Solving

Troubleshooting

Symptom	Possible cause - solution
Device doesn't work properly.	Diagnose the mobile lift.
Height adjustment doesn't work.	Replace main actuator.
The base legs doesn't spread.	Replace secondary actuator.
Problem with moving device.	Check/replace castors.

Diagnose the stand-up lift

In case of problems with height adjustment please complete following steps:

Step 1:

- Check the battery level on the hand control. If any led is lighting, please check the battery with the volt meter,
- If the battery is empty, please charge it,
- If after a few hours the battery is still empty, please check if the charger is working correctly by using the voltmeter. If the charger is ok, you have to replace the battery for a new one.

Step 2:

If battery level is correct, check the wires are connected properly to the control box in the bottom part of the device.

Checking Voltage on battery

Checking voltage on charger

Checking wires connection

seBastian II Service Manual

Step 3:

If everything looks good so far but the device still doesn't work, for further diagnose, spare parts will be necessary (hand control, control box and actuator).

- At first, replace the hand control for new one (or switch with another device if it's possible) and check,
- Next, prepare a new control box and connect it with slots from device,
- Now test what is wrong, control box, hand control or actuator.

Note – in case of situation that one of actuators is not working, make sure that problem is with actuator, by swapping the wires 1 and 2 for actuators in the control box and use the remote control to test (a working actuator means a faulty control box).

Step 4:

To remove control box, just loosen two main screws and lift up whole holder with control box.

Loosening two screws

Removing main holder

Replace main actuator

Step 1:

Use a flat screwdriver to remove the star lock cap and circlips from the upper rod holding the actuator (from only one side of the pin) and remove the pin. Do the same operation for the bottom side of the actuator.

Removing cover

Removing safety clip

Removing pin

Removing cover

Step 2:

Do the same operation for bottom side of the actuator.

Removing cover

Removing safety clip

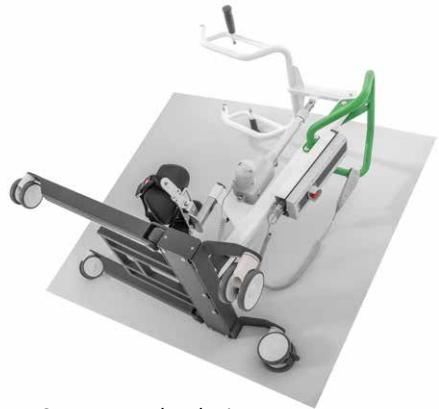
seBastian II Service Manual

Step 3:

Remove old and install the new actuator in the reverse order to dismantling, carefully tighten the gland.

Replace bottom actuator

Caution - Following procedures require turn down device on side. To protect it from any damage or scratches we strongly recommend to prepare some kind of protection (card boards, sponges, etc.) on the floor.

Sponges under device

Step 1:

Remove bottom and side metal covers.

Metal side cover

Removing screws from main cover

Step 2:

With remote control try to spread legs and check with actuator does not work.

Internal view

Step 3:

Remove only two screws from failure actuator's side (two top screws in that occasion), pull up actuator and remove bottom pin holding actuator.

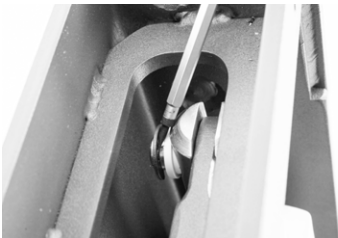
Unscrewing two screws

Removing safety ring

Step 4:

Find and remove safety ring from inside, the pull out holding pin.

Removal of the safety ring

Removing pin

Step 5:

Carefully take of the actuator from bottom frame.

View without pins

Actuator after take out

seBastian II Service Manual

Step 6:

Disconnect cable.

Disconnecting the cable

Step 7:

Remove old and install the new actuator in the reverse order to dismantling, carefully tighten the gland.

Wheel replacement

To replace rear wheels, use a socked wrench size 19". To replace front wheels, use an allen key and socked wrench size 17".

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