# HUMAN CARE

Your Life. Your Way.

## seBastian II Sit-to-stand Lift 90554

User manual / Användarmanual / Gebrauchsanweisung /  
Gebruikershandleiding

## Content

### ENGLISH

User manual	3
Intended use	4
Safety warnings	5
General maintenance and use	6
Safety instructions	7
Parts	10
Accessories	11
Technical specifications	12
Product label	14
Product symbols	14
Description of parts	15
Use	21
Maintenance instructions	23
Troubleshooting	26
Recycling	27

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---

## Important!

Always read the user manual before using your product and keep this manual for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the product itself.

**INFORMATION** symbol provides recommendations and information for the correct and easy use.

**SAFE WORKING LOAD** symbol indicates the maximum client weight for the product. Do not exceed this weight under any circumstances.

**CE MARKING** symbol indicates that the product is a medical device according to the definition of a medical device in EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product complies with the requirements of EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

**RECYCLE** according to local regulations.

**INSTRUCTIONS** do not use this product until you have read and understood the instructions.

***Product modification is not permitted. Always check before use that the product is not damaged and shows no signs of wear and tear.***

At Human Care, we continue to develop and improve our products in order to offer our customers the highest quality. We reserve the right

to make design changes without prior notice. Always make sure that you have the latest version of the user manual. It is available on our website, [www.humancaregroup.com](http://www.humancaregroup.com).

Please contact your local dealer if you have any questions about the product and its use.

# seBastian II

## Intended use

The seBastian II is an electric sit-to-stand lift, that allows for a natural movement when standing up whilst providing sufficient support.

The seBastian II is designed for active standing, sitting, dressing, toileting and balance training for clients with impaired trunk stability.

The seBastian II can be used to carry out the following lifting activities:

- from/to a chair or wheelchair
- from/to the toilet
- from/to a bed or stretcher
  
- Please read the instruction manual for both the lift and the lifting accessories before use. Lifting and transporting a person always involves a certain risk.
- It is important to fully understand the content of the manuals and to follow the instructions in the manuals.
- The product may only be used by trained personnel.
- In case of doubt or if you have any questions, please contact Human Care or your distributor.

### Do not use the product if:

The safe working load is exceeded.

The clips of the sling are not securely and properly attached.

The product is damaged.

You have not read and understood all the safety warnings and instructions.


## Safety warnings

Only trained professionals can make the decision whether or not this product is appropriate to use. This product can only be used under the supervision of a caregiver. Never leave the client unattended during the transfer process.

If you do not understand any of the warnings, safety precautions or instructions, please contact a care provider, distributor or technical staff member (if applicable) before attempting to use this product. Using this product without having the proper knowledge may result in injury or damage.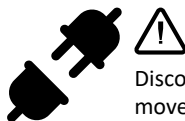

Disconnect the product and remove the battery during maintenance work.

Product modifications are not permitted. Maintenance and servicing may only be carried out by qualified personnel.

Only use the parts that are supplied with the product. Only use spare parts supplied by Human Care.

Maximum safe working load 230 kg/507 lbs

Do not touch parts of the products that may have an electrical current (leakage) at the same time as the patient.

## General maintenance and use

**Check the seBastian II before each use to make sure it is not damaged in any way. Do not use the seBastian II if there are signs of wear and tear or damage on any part of the lift.**

This product may only be used by qualified personnel.

The seBastian II may only be used on an even, clean and hard-floored surfaces that are free of obstacles. The product should never be used on a sloping floor or ramp.

In certain circumstances, more than one caregiver may be necessary to assist a particular client.

The client's abilities should be assessed before using this product. Only use this product for its intended use.

The seBastian II is designed to support only one client at a time. Never use the product to lift a client and an item of furniture at the same time.

Only use other Human Care products with the seBastian II.

This product is for indoor use only. It is not suitable for use in areas with high humidity (e.g. shower rooms, hydrotherapy), extreme temperature or high pressure environments.

A specialist must approve use of the product for a specific client and this must be recorded in writing.

Before using the product with a client, it is strongly recommended to practice using the product with colleague-caregivers, until the caregiver feels comfortable using the product.

The correct type, size, material and design of accessories must be carefully selected by an expert, taking into account the safety and needs of the client.

Always charge batteries in a well-ventilated room. Check the charger regularly.

## Safety instructions

- **Please read the instruction manual for both the lift and the lifting accessories before use. Lifting and transporting a person always involves a certain risk.**
- **It is important to fully understand the content of the manuals and to follow the instructions in the manuals.**
- **The product may only be used by trained personnel.**
- **In case of doubt or if you have any questions, please contact Human Care or your distributor.**

### Warning!

Certain conditions and surroundings may limit the use of the lift, e.g. thresholds, uneven surfaces, obstacles and deep-pile carpet. These conditions and surroundings may prevent the castors of the lift from working properly, may cause the lift to become unstable and/or may make it harder for the caregiver to use the lift. If you are not sure whether your care environment meets the requirements for proper use of the lift, please contact Human Care for help and advice.

### CAUTION!

This user manual contains important information for the user of this product. All users of the product must read and understand the content of the user manual and work accordingly. Always keep the user manual in a place where it is available to those persons using the product.

### CAUTION!

The assembled lift arm may only be raised by the lift actuator. Manual lifting the lift arm could damage the lift actuator.

Do not use the product in an environment where other radio frequency emitting devices are used. Like any other electronic device, the product can use and emit radio frequency energy. If the product is not used in accordance with the instructions in the user manual, it may cause interference with nearby devices. The manufacturer cannot guarantee that interference will not occur in a particular location. In order to check if the product is the source of any interference, change its location or disconnect the battery. If you still notice or suspect a fault, please try the following:

- Move the lift to another location;
- Increase the distance between devices;
- Contact the service-desk.

## General safety precautions

Always attach the sling according to the instructions for the sling that has been supplied.

Always check that the safe working load of the lift is suitable for the client's weight.

Always carry out lifting activities according to the instructions in the user manual.

Never disconnect a control or safety device and never try to by-pass such a function because you think it might make it easier to operate the lift.

Never attempt to move the lift by pushing against the mast, lift arm or client. Always use the appropriate push handle to move the lift.

When transporting a client, never move faster than a slow walking pace (3 kilometers per hour - 0.8 meters per second).

Do not push the lift over an uneven or rough surface, especially when transporting a client.

Do not try to push/pull a loaded lift over an obstacle on the floor, which the castors cannot clear.

Never force a control/safety device. All controls are easy to use and do not require the use of physical force.

Never leave a loaded lift on a sloping surface.

Never use an electric lift in the shower.

Never take the lift down a flight of stairs, either with or without a client.

Before moving the lift, always check that it is disconnected from the charger, as this could damage the electrical connection.

### Always make sure you follow these instructions before using the lift:

Always carry out a daily check before using the lift;

Always read the instruction manuals for the lift and the lifting accessories. You know what you need to do before starting;

The caregiver should always familiarize himself/herself with the controls and safety devices before attempting to lift a client;

Only use slings that are recommended for use with this lift.

Ensure that the correct lifting accessories are being used in terms of type, size, material and design to meet the client's needs;

Make sure the lift accessories are not damaged;

Ensure that the brakes are intact; any missing or damaged brakes must always be replaced;

The lifting accessories are correctly attached to the lift;

Ensure that the clips of the sling are correctly clipped to the hanger bar after the sling has been fully extended, but before the client has actually been lifted;

Ensure the batteries are fully charged.

# seBastian II

## Parts

- 1. Hand control**  
Controls the functions of the lift.
- 2. Push handle**  
Handle for the caregiver to move the lift.
- 3. Lift arm**  
Raises/lowers the client in a sitting position, always in combination with a sling.
- 4. Control box with emergency stop and battery**  
Power pack.
- 5. Lift actuator**  
Electric lift actuator for raising and lowering the lift arm.
- 6. Height-adjustable lower leg supports**  
Soft, comfortable lower leg supports for the client's shins.
- 7. Castors, rear**  
Double castors, with brake
- 8. Electric leg spreading**  
Provides the option to position the legs around a chair or wheelchair, so that the lift can be positioned closer to the client in order to achieve a more comfortable lifting process.
- 9. Castors, front**  
Double castors, without brake.
- 10. Chassis**  
Chassis with four castors.
- 11. Foot plate**  
Plate for the client's feet to stand up safely.

## Accessories

2-point hanger bar for E-clip slings:

Product name	Article number
Back Belt E-clips Small	25130C-S
Back Belt E-clips Medium	25130C-M
Back Belt E-clips Large	25130C-L
Back Belt E-clips Extra Large	25130C-XL










All slings are available in different sizes to achieve a best possible fit with the client.

Product name	Article number
Soft padding for the bar	50866

## Technical specifications

Description	
Article number	90554
Length of the frame	1080 mm / 42.52"
Width of the frame	750 mm / 29.53"
Height	1130 mm / 44.49"
Position of the push handle (min/max)	950 - 1130 mm / 37.4 - 44.49"
Height of the lower leg supports, mounted downwards (min- max)	390- 520 mm / 15.35 - 20.47"
Height of the lower leg supports, mounted upwards (min- max)	530- 660 mm / 20.87- 25.98"
Lifting height (min.)	780 mm / 30.71"
Lifting height (max.)	1740 mm / 68.5"
Minimum width of legs (internal/external)	520 mm / 20.47" / 660 mm / 25.98"
Maximum width of legs (internal/external)	1010 mm / 39.76" / 1150 mm / 45.28"
Height of legs (from the floor)	115 mm / 4.53"
Footplate, height	90 mm / 3.54"
Footplate, dimensions (wxd)	360 x 330 mm / 12.99 x 14.17"
Turning radius (min.)	1240 mm / 48.82"
Turning radius (max.)	1400 mm / 55.12"
Lifting speed (up)	18 sec.
Lifting speed (down)	18 sec.
Diameter of castors with brake	125 mm / 4.92"
Diameter of castors without brake	100 mm / 3.94"
Operation	continuous, short-term load (10%) max. 2 min. in operation (on) min. 18 minute break (off)
Control box	input voltage 100-240V ~, frequency 50/60 Hz, output voltage 24V, power consumption max. 400 mA
Battery capacity	2.9 Ah
Electric shock protection class	II, adapted part type B
IP rating control box	IPX4
IP rating class battery	IPX5
IP rating main lift actuator	IPX4
IP rating hand control	IPX6
Maximum user weight	230 kg / 507.06lbs
Product weight	60 kg / 132.28lbs
Noise level	52 dB
Frame material	Powder-coated steel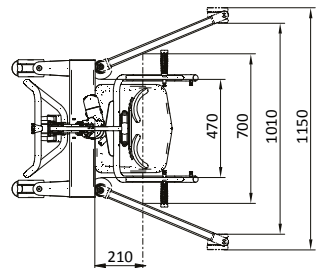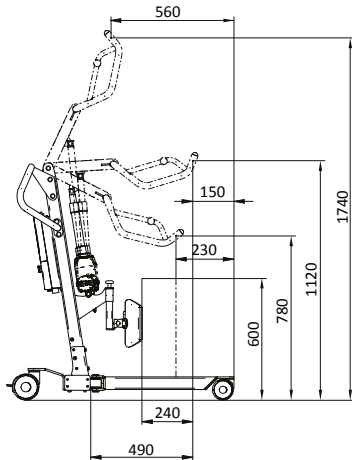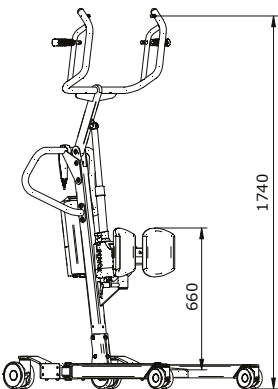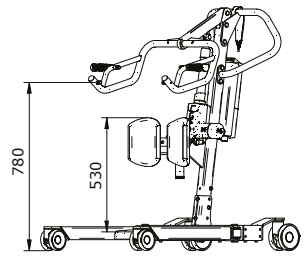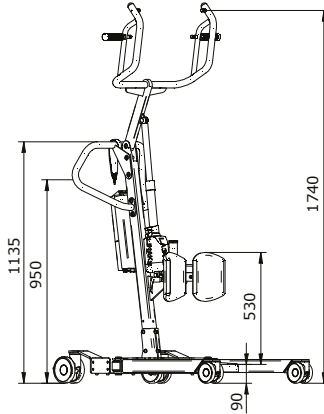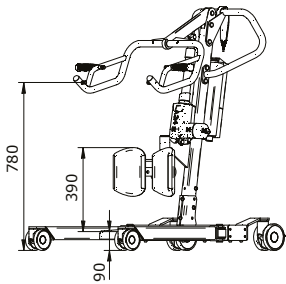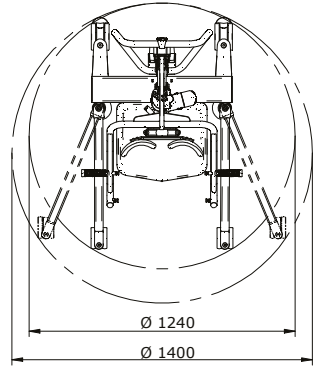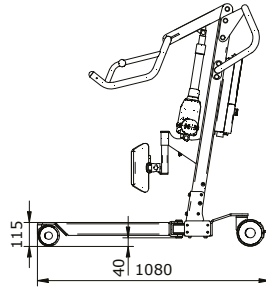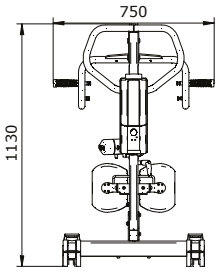

## Product label


The product label contains information about the product and is located on the right-hand side, close to the mast.

Human Care is an ISO 13485:2016 Swedish certified medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product has been tested in accordance with ISO 10535.

The product has a CE marking in accordance with the Medical Device Directive 93/42/EEC and/or Medical Device Regulation (EU 2017/745). The CE mark label is at the product.

This product has been tested by an accredited test institution and meets the following necessary medical requirements: EN 10535, EN 60601-1 (electrical safety) and EN 60601-1-2 (EMC). This product carries the CE marking.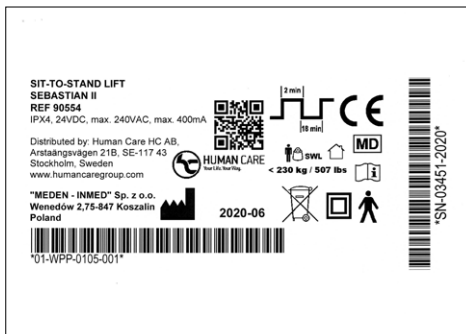

## Product symbols

Symbols on the product and the product label:

	Product operating cycle
	CE marking
	MD Medical Device
	Product manufacturer
	Maximum user weight

	For indoor use
	User manual: Please read the manual to familiarize yourself with its contents before using the product
	This product contains electrical and electronic components and must be recycled according to local guidelines.
	Electric shock protection class II
	Product type B: approved for physical contact with clients (IEC 60601-1)

## Description of parts

### Hand control

The seBastian II sit-to-stand lift has a hand control with the following functions:

#### 1. Battery charge

The battery charge indicator has three status indications. The battery charge indicator is yellow or green until switched off (2 minutes after use).

#### Battery status

For a complete description we refer to page 18.

#### 2. Service indicator

The service indicator (yellow indicator) flashes when it is time to carry out a service. The default setting is every 12 months / 8000 lifting cycles (as recommended by the EN10535 standard), whichever comes first. When it is time for a service, the service indicator stays on for 2 minutes after use. It is then switched off to save battery life. The system is still functional when the service indicator is on.

#### 3. Overload indicator


When there is an overload, according to the predefined power interruption, the two LED lights will flash for 10 seconds.

#### 4. Up

This button raises the lift arm, to help a client into a standing-up position.

#### 5. Down

This button lowers the lift arm, to help a client into a seated position.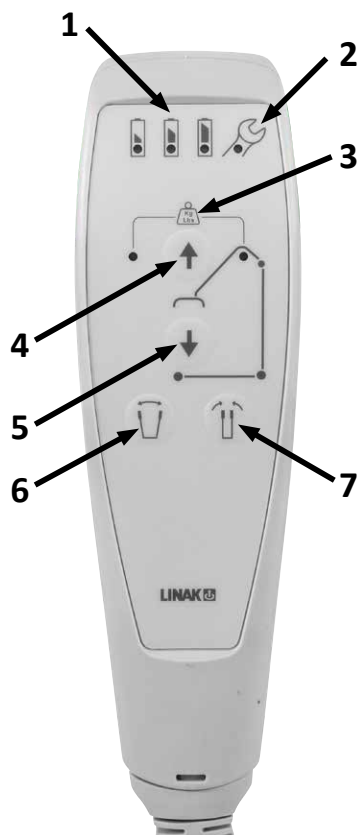

#### 6. Leg adjustment, open

The legs can be spread so that the lift can be brought closer to a client sitting in a chair for a more natural lifting movement.

#### 7. Leg adjustment, close

The legs can be closed to make it easier to maneuver and negotiate narrow passageways (e.g. doorways).

## Emergency stop

In the event of an emergency, immediately press the emergency stop button.

Turn the button clockwise to unlock the emergency stop.

## Electric emergency lowering function

If the button on the hand control for lowering the lift arm does not work, you can use the electric emergency lowering function. This can be activated by using a pointed object (e.g. a pen) to press the small button on the control box. You will hear an audible click. Keep pressing until the required height has been reached.

## Mechanical emergency lowering function



If the emergency lowering function via the control box does not work, use the mechanical lowering function. This will safely lower the client in the event of a fault in the electrical system. Lowering can be done manually by using the lever located on the lower part of the main cylinder. To operate, grasp the red part and pull it up. Allow the lift arm to lower until the user can be safely removed from the lift.

## Caution!

After using the mechanical or electrical emergency lowering function, you must immediately contact your distributor to schedule a visit by a service technician. The product may only be used again after an authorized employee has carried out an inspection of the product.

Electrical emergency lowering

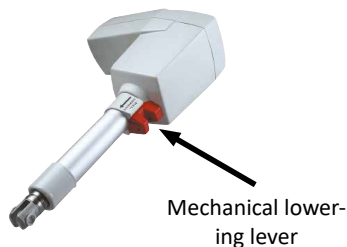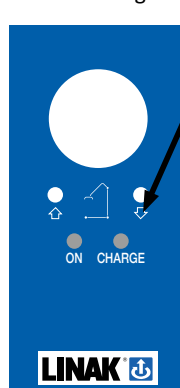## Battery

### Installing and removing the battery


#### Caution!

Do not use the lift for at least one hour before replacing the battery.

Never expose the battery to extreme temperatures. Never leave the battery charger in damp/humid areas.


#### Installing the battery:

- Align the bottom of the battery with the top of the controller.
- Push the top of the battery forward until you hear a click. The battery is now in place.
- Gently push the top of the battery forward to make sure it is securely attached to the holder.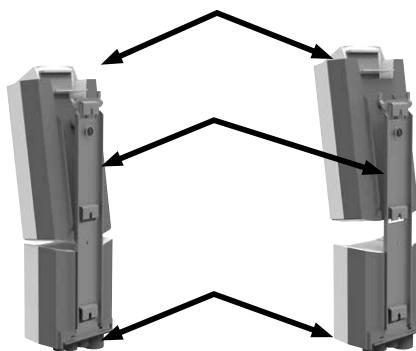

#### Removing the battery

- Press the emergency stop button.
- Hold the battery by the handle and press the release lever
- Remove the battery by lifting it off the holder

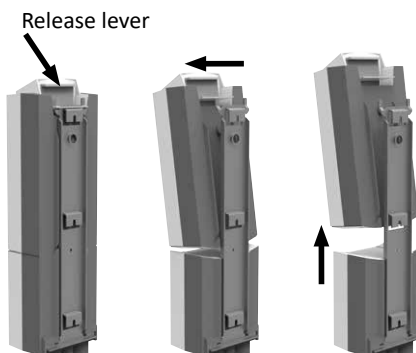## Battery charge display and charging

### Caution!

- The product cannot be used while charging.
- During charging, the lifting function is locked.
- Do not use the lift for 1 hour after charging. This will prolong the life of the battery.
- The battery needs to be charged continuously for at least 24 hour, in the following situations:
  - a) Before using the lift for the first time.
  - b) Before being placed in storage (up to 3 months) without a power supply connection.
  - c) Immediately after a storage period.

## Battery charge indicator

The battery charge indicator has three status indications. The battery indicators are either yellow or green until switched off (2 minutes after use).

### Battery status 1:

The battery is full – no recharging required (100-50%): the third indicator is green.

### Battery status 2:

The battery is okay (50-25%) – recharging required: the second indicator is yellow.

### Battery status 3:

The battery is low (less than 25%) – recharging required: the first indicator is yellow and a buzzer sounds whenever a button is pressed.





### Battery status 4:

The battery requires recharging. Some functions of the lift are lost and it is only possible to lower the lift arm. When in continued use, the indicator flashes and an audio signal is emitted.

The battery charge is indicated on the hand control.


Battery level indicators on the hand control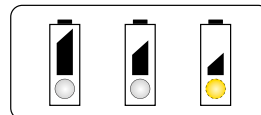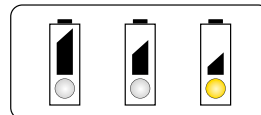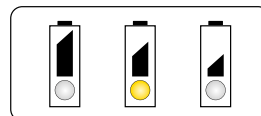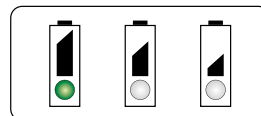

## Charging the battery

If you need to charge the battery, you need to:

- Press the emergency stop button
- Connect the black charging cable that is supplied to the control box
- Then insert the plug into a wall socket that is located in a dry room.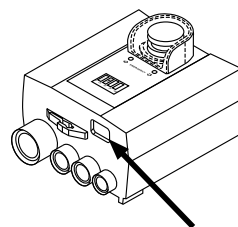

Connection for charging cable

When the rechargeable battery is connected, the LED display on the control box will come on.

The green LED indicates that the battery charger is connected to the mains. The yellow LED indicates the charging process. It takes approximately 5 hours to fully charge the battery.

## Charging the battery with an external charger (optional).



You can always purchase an external charger, for example to have a spare battery to hand. To charge the battery, it needs to be inserted in the external charging station. Ensure that the battery is properly inserted into the charging station. A light will come on if it is attached correctly.

If the battery emits a short beep, it needs to be charged. If a green light comes on when the battery is inserted, the battery is full. If the orange light comes on, the battery needs to be charged. Charging will stop automatically when the battery is full. This means you cannot overcharge the battery. It is best to charge the battery every night.

The external charger can be easily mounted by means of the wall mounting plate. It can be mounted on the wall near a 230 V socket.

We always recommend you purchase a spare battery, so that you always have access to a full battery. If one battery is empty, you can continue to work using the second battery.

Location of LED on control box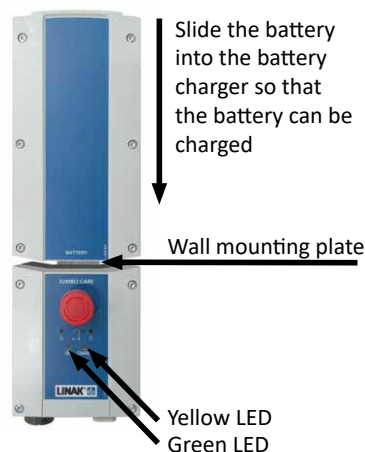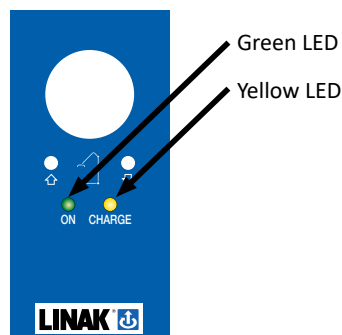


Charging the battery must not take place in a wet/humid environment (shower area).

# seBastian II

## Leg adjustment

This function is useful when you need to move the lift to a client on a chair or other piece of furniture. The width of the chassis can be adjusted, so that the legs of the lift can fit around furniture. This means that the lift arm and the footplate are closer to the client, which encourages a natural movement when standing up. Press and hold the corresponding button on the hand control until the legs are in the correct position.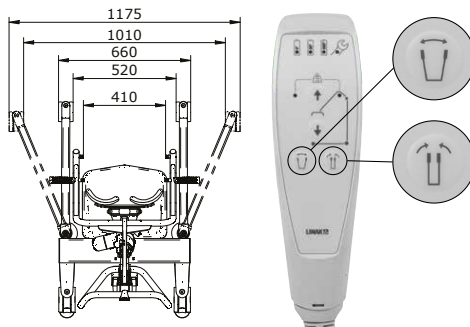


## Brakes on the castors

The brakes are important components of the lift. The brakes are mounted on the rear two castors and prevent unwanted movement during use of the lift to ensure a stable starting position.

The brake is applied by using your foot to push down the red lever.


The brake is released by using your foot to push up the red lever.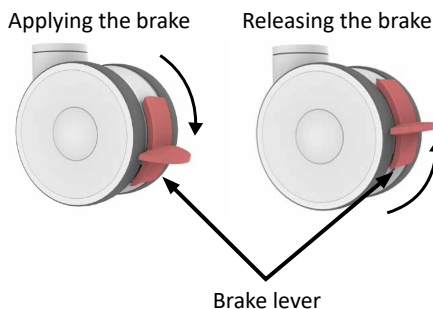

## Lower leg supports


Adjusting the height of the lower leg supports

- Turn the clamping knob anti-clockwise to release the lower leg supports.
- Pull the lower leg supports up or down until they are in the correct position.
- Turn the locking button clockwise to lock the lower leg supports.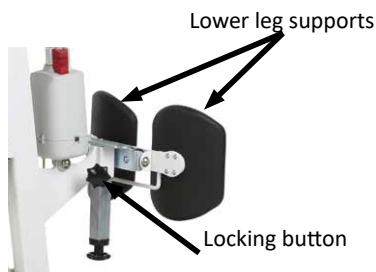

Making further adjustments to the leg supports

- Pull and hold the locking button.
- Adjust the level and release the locking button.

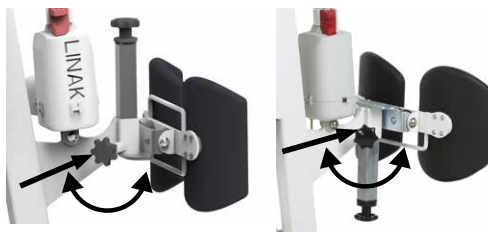## Use

### Moving the seBastian II

The push handle must be used for moving and maneuvering the seBastian II.

Think about your posture:

- Walk behind the lift when pushing straight ahead.
- Start moving the lift by using your own body weight as much as possible.
- Keep your back as straight as possible.
- When you need to make a turn, keep one hand on the push handle and use your other hand to slightly push against the lift arm.




Push handle








### Standing up

To support the client in standing up using the sit-to-stand lift, the following steps are recommended:

1. Lower the lift arm as much as possible (lift arm button, make sure it does not touch the client)
2. Make sure that the client is in a safe, seated position
3. Position the sling, Back Belt 25130C, around the waist of the client
4. Place the lift around the client's chair so that the client is able to place his/her feet on the footplate
5. Lock the brakes of the other device, like a chair or bed
6.  Ensure the brakes are not engaged when lifting the client. The lift should be able to move to its centre of gravity.
7. If possible, the client should hold on to the handles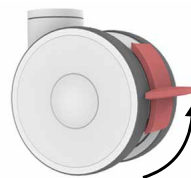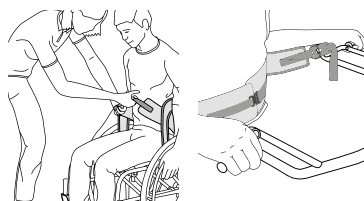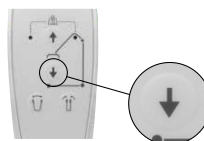

- Adjust the lower leg supports to the correct height, approximately 2 cm below the client's knees. If necessary, bring the lift closer by opening the legs more (button 1 on the hand control).
- Attach the sling to the connection points of the lift arm without using excessive force. Carefully check each of the attachments. Using the hand control, raise the lift arm until there is some tension in the sling but do not start lifting the client yet. Check the attachments again.
- Press the up button to move the client to the desired standing position.
- Move the lift backwards so that the legs can be closed again (2nd button on the hand control).
- Start moving the client. For the comfort of the client it is better to keep to short distances.
- Once you have reached your destination, apply the brakes again. If necessary, open the legs of the lift before getting close to another object (e.g. chair/toilet).
- Lower the client by holding down the relevant down button until the client is seated and the sling can be released.
- Remove the sling after you have made sure that the client is fully seated on a firm and stable surface, with support for the feet.
- Move the product away and park it in a safe place.
- When the lift is parked, the brakes of the castors can be applied.
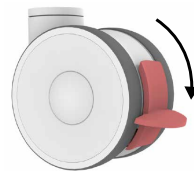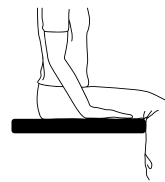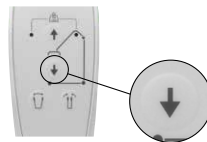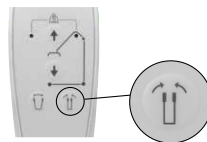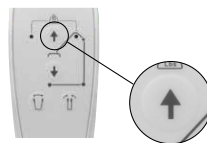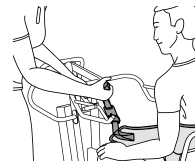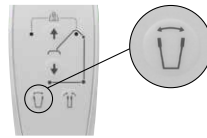
## Maintenance instructions

### Storage and transport

The product must be kept in a cool and dry room. The climate conditions of the room must be within the following parameters:

- Temperature 10-40 °C (recommended 20 °C or lower);
- Humidity 30 ÷ 75%;
- Air pressure: 700 ÷ 1060 hPa.

Do not use the lift in environments where flammable materials may be present, e.g. in storage areas for flammable substances.

### Cleaning

Cleaning is an important requirement to ensure that the lift works properly over a long period of time. Routine cleaning of the lift will suffice if it is used by the same client.

The lift will need to be disinfected if there are visible signs of material or possibly infected material (blood, stool, pus) or in situations requiring a high level of high infection control.

#### CAUTION!

The seBastian II must be cleaned and disinfected every time a different client uses the lift before it can be used again.

#### CAUTION!

Before cleaning the lift, check that:

- All plugs are correctly connected.
- The battery is mounted on top of the control box.
- None of the electrical components show external damage. Failure to do this may result in water or cleaning agents to get into the lift and cause interference with the unit or damage to electrical components.
- Electrical components cannot be washed with water jets or high-pressure cleaners, etc. They can only be cleaned with a damp cloth.
- Stop using the lift if you suspect that water or liquid has entered the electrical components and report the incident immediately to the service center.
- Failure to comply with the above rules may result in serious damage to the equipment and could have other unforeseen consequences.

## Cleaning contact surfaces

1. The surface of the handles must be cleaned and maintained as follows:
  - a) undo the straps and remove all parts that do not belong to the lift.
  - b) clean surfaces with mild and environmentally-friendly cleaning agents
  - c) the same applies to cleaning the hand control.
2. Accessible areas of the push handles and the frame can be disinfected using mild and environmentally-friendly detergents
3. The castors only need to be disinfected if there are visible traces of infected or potentially contaminated material.

### Do not use:

- pastes, waxes, sprays,
- strong detergents, solvents and detergents containing solvents, alcohol and leather cleaners.


Using such agents can also cause hardness and cracks in the material. Any changes to the surface sheen are not covered under the warranty!

## Servicing

An annual service and maintenance by authorized personnel is required every 12 months.


Human Care cannot be held responsible for damage caused by willful damage, overdue maintenance and incorrect use, or any resulting consequential damage.

We recommend checking the product regularly for loose parts. Regularly clean the castors.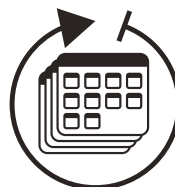

## Maintenance

During long periods of inactivity, we recommend checking all electrical and mechanical parts once a month by carrying out a lift test without the client. Additionally, the charging cables should be visually checked for possible damage after each mechanical lift or after moving the lift to a different location.

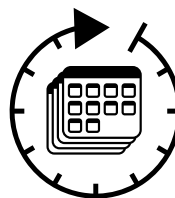## Maintenance of the frame

The metal sections of the frame can be cleaned with a soft, damp cloth. Cleaned surfaces should be wiped dry each time. Do not use cleaning products containing alcohol.

All moving joints must be lubricated once every six months or when they start making a loud noise when used. The moving joints include:

- castors and actuators;
- bearing bushes and joints of the legs and lift arm.

As a lubricant, we recommend using commercially available penetrating and lubricating products (e.g. Wurth HHS 2000). Any spillage of such lubricants must be removed immediately with a dry cloth.

Threaded connections should be inspected periodically – once every six months – and, if necessary, any accidental tolerance must be remedied. Any inadvertent tolerances must be reported to the manufacturer's service center. The product can no longer be used until the cause of this tolerances has been removed.

## Maintenance of the frame

A service is required every 12 months or 8000 lifting cycles, whichever comes first and every time after a fault/repair of the lift. Services, including the electrical safety test, must be carried out by a certified person.

Services should include as a minimum:

Check whether there is mechanical damage to the wiring;

Check whether there is mechanical damage to the housing of the controller, battery and actuator;

Check whether the height adjustment and leg adjustment buttons are working properly.

## Troubleshooting

Problem	Solution
The lift is not responding to the hand control.	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the hand control cable is connected
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the other wires are connected
	Check whether the service LED is flashing
	Check whether the lift responds to the raising/lowering button on the controller
	Contact the service department
The lift is not responding to the control box.	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the other wires are connected
	Check whether the service LED is flashing
	Contact the service department
The lift is not charging	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the service LED is flashing
	Contact the service department
The lift stops while lifting the client.	Check whether the overload indicator is flashing
	Check whether the battery is sufficiently charged
	Check whether the service LED is flashing
	Check whether the lift responds to the raising/lowering button on the controller
	Manually lower the lift
	Contact the service department
The lift is making strange noises (cracking, crossover, etc.).	Contact the service department
The lift cannot be moved	Check whether the brakes on the rear castors are locked
	Contact the service department

If the error messages persist, stop using the lift immediately and contact the distributor for further instructions.

## Recycling

The seBastian II is an electronic appliance covered by Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). The product label shows the WEEE symbol, which consists of a crossed-out wheeled bin. This label indicates that the batteries must be collected separately at the end of the product's lifecycle. This requirement applies to the European Union. Do not dispose of these products in household waste.

### Caution!

Batteries and electronics contain lead and must be recycled according to WEEE 2002/96/EC.

Metals

Electrical and electronic waste (WEEE)

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