HUMAN CARE

Your Life. Your Way.

Diana II Comfort 4MSB 90559 /
Diana II Comfort 4ESB 90560

User manual / Bruksanvisning / Gebrauchsanweisung /
Gebruikershandleiding

Content

ENGLISH

User manual	3
Intended use	4
Safety warnings	5
General maintenance and use	6
Safety instructions	7
Parts	10
Accessories	11
Technical specifications	12
Product label	14
Product symbols	14
Description of parts	15
Use	22
Maintenance instructions	25
Troubleshooting	28
Recycling	29

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User manual

Important!

Always read the user manual before using your product and keep this manual for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the product itself.

INFORMATION symbol provides recommendations and information for the correct and easy use.

SAFE WORKING LOAD symbol indicates the maximum client weight for the product. Do not exceed this weight under any circumstances.

CE MARKING symbol indicates that the product is a medical device according to the definition of a medical device in EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product complies with the requirements of EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

RECYCLE according to local regulations.

INSTRUCTIONS do not use this product until you have read and understood the instructions.

Product modification is not permitted. Always check before use that the product is not damaged and shows no signs of wear and tear.

At Human Care, we continue to develop and improve our products in order to offer our customers the highest quality. We reserve the right

to make design changes without prior notice. Always make sure that you have the latest version of the user manual. It is available on our website, www.humancaregroup.com.

Please contact your local distributor if you have any questions about the product and its use.

Diana II Comfort

Intended use

The Diana II Comfort is an electrically operated lift that provides support to clients while they are lifted from their bed, in/out of a chair or wheelchair, on/off the toilet and even from the floor. The Diana II Comfort is a stable and compact lift with a high maximum user weight, also allowing heavier people to be safely supported when they are being lifted. Our extensive range of E-clip slings provide the required support.

This lift is suitable for many types of clients and lifting situations, due to the large range of the lift arm and the adjustable 4-point spreader bar. Using the 4-point spreader bar allows the caregiver to place the client into the desired position, varying from a recumbent to a sitting position. A practical solution is the manually operated spreader bar: it can provide three positions: recumbent, semi-recumbent and sitting. For frequent use, we recommend an electric 4-point spreader bar, where every angle inbetween recumbent and sitting is a possibility.

It is easy to maneuver the lift thanks to the large, high-quality twin castors. The red pedal is used to easily and quickly operate the brakes on the castors.

The electric leg operation allows the lift to be placed around a chair/toilet to get as close

as possible to a client for a more comfortable transfer.

Optimal support can be provided by combining the Diana Comfort II with one of our slings. Unique are our E-clips for attaching the slings. The universal E-clip is a proven system for fastening a sling to a lift, thereby significantly reducing the risk of error. The clip is placed around the snap hooks of the hanger bar and closes with an audible 'click'. It is a signal to the caregiver that the clip is securely attached and cannot unexpectedly become loose.

Do not use the lift if:

The safe working load is exceeded.

The clips of the sling are not securely and properly attached.

The product is damaged.

You have not read and understood all the safety warnings and instructions.

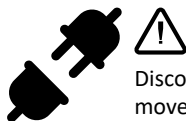
Safety warnings

Only trained professionals can make the decision whether or not this lift is appropriate to use. This product can only be used under the supervision of a caregiver. Never leave the client unattended while in the lift.

If you do not understand any of the warnings, safety precautions or instructions, please contact a care provider, distributor or technical staff member (if applicable) before attempting to use this product. Using this product without having the proper knowledge may result in injury or damage.

Disconnect the product and remove the battery during maintenance work.

Product modifications are not permitted. Maintenance and servicing may only be carried out by qualified personnel.

Only use the parts that are supplied with the product. Only use spare parts supplied by Human Care.

Product modifications are not permitted. Maintenance and servicing may only be carried out by qualified personnel.

Do not touch parts of the products that may have an electrical current (leakage) at the same time as the patient.

Maximum safe working load
230 kg/507 lbs

Diana II Comfort

General maintenance and use

Check the Diana II Comfort before each use to make sure it is not damaged in any way. Do not use the Diana II Comfort if there are signs of wear and tear or damage on any part of the lift.

This lift may only be used by qualified personnel.

The Diana II Comfort may only be used on an even, clean and hard-floored surfaces that are free of obstacles. The lift should never be used on a sloping floor or ramp.

In certain circumstances, more than one caregiver may be necessary to assist a particular client.

The client's abilities should be assessed before using this product. Only use this product for its intended use.

The Diana II Comfort is designed to support only one client at a time. Never use the lift to lift a client and an item of furniture at the same time.

Only use other Human Care products with the Diana II Comfort.

This lift is for indoor use only. It is not suitable for use in areas with high humidity (e.g. shower rooms, hydrotherapy), extreme temperature or high pressure environments.

A specialist must approve use of the lift for a specific client and this must be recorded in writing.

Before using the lift for a client, it is strongly recommended to practice using the lift with colleague-caregivers, until the caregiver feels comfortable using the product.

The correct type, size, material and design of accessories must be carefully selected by an expert, taking into account the safety and needs of the client.

Always charge batteries in a well-ventilated room. Check the charger regularly.

Safety instructions

- **Please read the instruction manual for both the lift and the lifting accessories before use. Lifting and transporting a person always involves a certain risk.**
- **It is important to fully understand the content of the manuals and to use the lift as per the instructions in the manuals.**
- **The lift may only be used by trained personnel.**
- **In case of doubt or if you have any questions, please contact Human Care or your distributor.**

Warning!

Certain conditions and surroundings may limit the use of the lift, e.g. thresholds, uneven surfaces, obstacles and deep-pile carpet. These conditions and surroundings may prevent the castors of the lift from working properly, may cause the lift to become unstable and/or may make it harder for the caregiver to use the lift. If you are not sure whether your care environment meets the requirements for proper use of the lift, please contact Human Care for help and advice.

CAUTION!

This user manual contains important information for the user of this product. All users of the lift must read and understand the content of the user manual and work accordingly. Always keep the user manual in a place where it is available to those persons using the product.

Diana II Comfort

General safety precautions

Always attach the sling according to the instructions for the sling that has been supplied.

Always check that the safe working load of the lift is suitable for the client's weight.

Always carry out lifting activities according to the instructions in the user manual.

Never disconnect a control or safety device and never try to by-pass such a function because you think it might make it easier to operate the lift.

Never attempt to move the lift by pushing against the mast, lift arm or client. Always use the appropriate push handle to move the lift.

When transporting a client, never move faster than a slow walking pace (3 kilometers per hour - 0.8 meters per second).

Do not push the lift over an uneven or rough surface, especially when transporting a client.

Do not try to push/pull a loaded lift over an obstacle on the floor, which the castors cannot clear.

Never force a control/safety device. All controls are easy to use and do not require the use of physical force.

Never leave a loaded lift on a sloping surface.

Never use an electric lift in the shower.

Never take the lift down a flight of stairs, either with or without a client.

Before moving the lift, always check that it is disconnected from the charger, as this could damage the electrical connection.

Always make sure you follow these instructions before using the lift:

Always carry out a daily check before using the lift;

Always read the instruction manuals for the lift and the lifting accessories. Determine what you need to do before you start using the lift;

The caregiver should always familiarize himself/herself with the controls and safety devices before attempting to lift a client;

Only use slings that are recommended for use with this lift.

Ensure that the correct lifting accessories are being used in terms of type, size, material and design to meet the client's needs;

Make sure the lift accessories are not damaged;

Ensure that the brakes are intact; any missing or damaged brakes must always be replaced;

The lifting accessories are correctly attached to the lift;

Ensure that the clips of the sling are correctly clipped to the hanger bar after the sling has been fully extended, but before the client has been actually lifted.

Ensure the batteries are fully charged;

Radio frequency

Do not use the lift in an environment where other radio frequency emitting devices are used. Like any other electronic device, the lift can use and emit radio frequency energy. If the lift is not used in accordance with the instructions in the user manual, it may cause interference with nearby devices. The manufacturer cannot guarantee that interference will not occur in a particular location. In order to

check if the lift is the source of any interference, change its location or disconnect the battery. If you still notice or suspect a fault, please try the following:

- Move the lift to another location;
- Increase the distance between devices;
- Contact the service-desk.

Diana II Comfort

Parts


- 1. Push handle**
Handle for the caregiver to move the lift.
- 2. Hand control**
Controls the functions of the lift.
- 3. Control box with emergency stop and battery**
Power pack.
- 4. Castors, rear**
Double castors with brake.
- 5. Electric leg spreading**
Provides the option to position the legs around a chair or wheelchair, so that the lift can be positioned closer to the client in order to achieve a more comfortable lifting process..
- 6. Castors, front**
Double castors without brake.

- 7. Chassis**
Chassis with 4 castors.
- 8. MSB: Mechanical Spreader Bar**
Can be used to manually position the client in 3 positions: recumbent, semi-recumbent and sitting.
ESB: Electric spreader bar, including lift actuator and rotation lock.
- 9. Lift actuator**
Electric lift actuator for raising and lowering the lift arm.
- 10. Lift arm**
Raises/lowers the client in a sitting position, always in combination with a sling.

Accessories

E-clip slings

Product name	Article number
Full Body Sling E-clip Net	25005C-(size)
Full Body Sling E-clip Soft	25010C-(size)
Columbus Sling E-clip Soft	25040C-(size)
Toileting Sling High E-clip	25090C-(size)

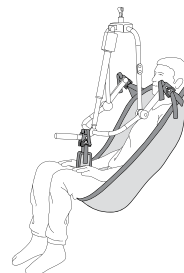All slings are available in different sizes to achieve a best possible fit with the client.



Product name	Article number
Soft padding for the bar	50866

Diana II Comfort


Technical specifications

Description	
Article number	90559 (4MSB), 90560 (4ESB)
Length of the frame	1300 mm / 51.2"
Width of the frame	680 mm / 26.8"
Height	1330 mm / 52.4"
Height of the push handle	950-1270 mm / 37.4-50"
Lifting height (min.)	650 mm / 25.6"
Lifting height (max.)	1920 mm / 75.6"
Minimum width of legs	520 mm / 20.5"
Maximum width of legs	1180 mm / 46.6"
Height of legs (from the floor)	115 mm / 4.5"
Angle of the 4-point spreader bar	13°, 29°, 45° (MSB), -17° +42°(ESB)
Turning radius (min.)	1440 mm / 56.7"
Turning radius (max.)	1580 mm / 62.2"
Lifting speed (up)	36 sec.
Lifting speed (down)	34 sec.
Diameter of castors with brake	125 mm / 4.9"
Diameter of castors without brake	100 mm / 3.9"
Operation	continuous, short-term load (10%) max. 2 min. in operation (on) min. 18 minute break (off)
Control box	input voltage 100-240V ~, frequency 50/60 Hz, output voltage 24V, power consumption max. 400 mA
Battery capacity	2.9 Ah
Electric shock protection class	II, adapted part type B
IP rating control box	IPX4
IP rating class battery	IPX5
IP rating main lift actuator	IPX4
IP rating hand control	IPX6
Maximum working load	230 kg / 507lb
Product weight	66 kg / 145lb
Noise level	52 dB
Frame material	Powder-coated steel

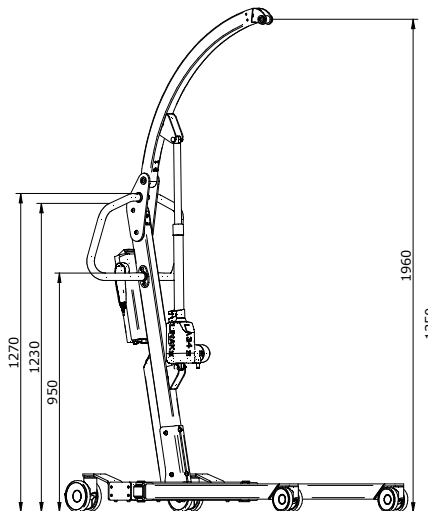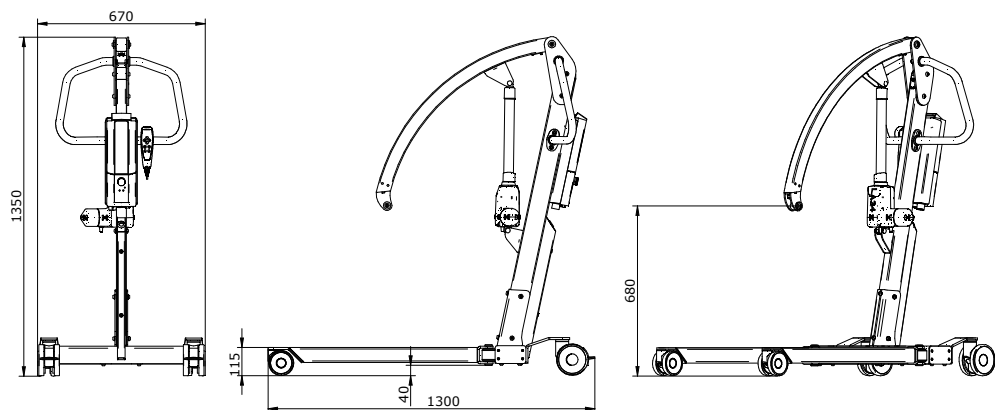Front and side view

Turning radius

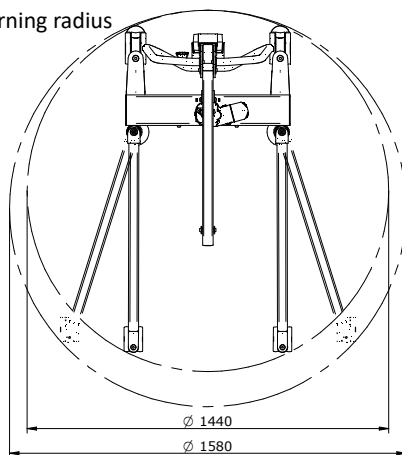Diana II Comfort

Product label

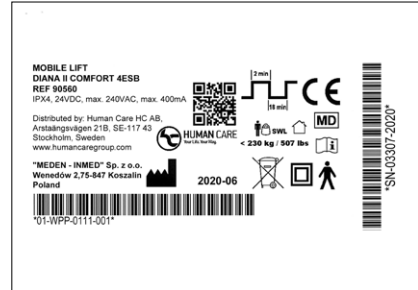
The product label contains information about the lift and is located on the right-hand side, close to the mast.

In compliance with / Standards / CE

Human Care is an ISO 13485:2016 Swedish certified medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product has a CE marking in accordance with the Medical Device Directive 93/42/EEC and/or Medical Device Regulation (EU) 2017/745). The CE mark has been affixed to the lift.

This product has been tested by an accredited test institution and meets the necessary medical requirements and is in accordance with medical requirements: EN 10535, EN 60601-1 (electrical safety) and EN 60601-1-2 (EMC). This product carries the CE marking.

Product symbols

Symbols on the product and the product label:

	Product operating cycle
	CE marking
	MD Medical Device
	Product manufacturer
	Maximum working load

	For indoor use
	User manual: Please read the manual to familiarize yourself with its contents before using the product
	This product contains electrical and electronic components and must be recycled according to local guidelines.
	Electric shock protection class II
	Product type B: approved for physical contact with clients (IEC 60601-1)

Description of parts

Hand control

The hand control allows you to control the various functions of the lift.

Depending on the version, two types of hand control are available.

1. Battery charge

The battery charge indicator has three status indications. The battery charge indicator is yellow or green until switched off (2 minutes after use).

For a full description, see page 18.

2. Service indicator

The service indicator (yellow indicator) flashes when it is time to carry out a service. The default setting is every 12 months / 8000 lifting cycles (as recommended by the EN10535 standard), whichever comes first. When it is time for a service, the service indicator stays on for 2 minutes after use. It is then switched off to save battery life. The system is still functional when the service indicator is on.

3. Overload indicator

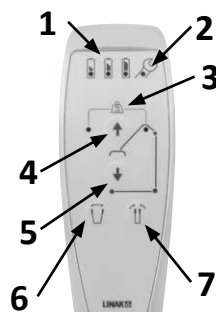
When there is an overload, according to the predefined power interruption, the two LED lights will flash for 10 seconds.

4. Up


This button raises the lift arm, to help a client into a standing-up position.

5. Down

This button lowers the lift arm, to help a client into a seated position.

Diana II Comfort MSB

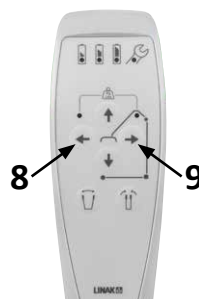Diana II Comfort ESB

6. Leg adjustment, open

The legs can be opened so that the lift can be brought closer to a client sitting in a chair or lying on the floor. This text is also valid for the Diana II.

7. Leg adjustment, close

The legs can be closed to make it easier to maneuver and negotiate narrow passages (e.g. doorways).

8. Tilting position, forwards

9. Tilting position, backwards

Diana II Comfort

Emergency stop

In the event of an emergency, immediately press the emergency stop button.

Turn the button clockwise to unlock the emergency stop.

Electric emergency lowering function

If the button on the hand control for lowering the lift arm does not work, you can use the electric emergency lowering function. This can be activated by using a pointed object (e.g. a pen) to press the small button on the control box. You will hear an audible click. Keep pressing the button until the required height has been reached.

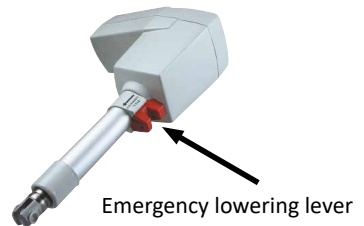
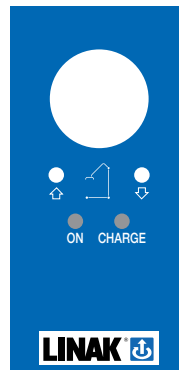
Mechanical emergency lowering function

If the emergency lowering function via the control box does not work, use the mechanical lowering function. This will safely lower the client in the event of a fault in the electrical system. Lowering can be done manually by using the lever located on the lower part of the cylinder just above the control box. To operate, grasp the red part and pull it up. Allow the lift arm to lower until the user can be safely removed from the lift.

Caution!

After using the mechanical or electrical emergency lowering function, you must immediately contact your supplier, distributor or municipality to schedule a visit by a service technician. The lift may only be used again after an authorized employee has carried out an inspection of the lift.

Battery

Installing and removing the battery

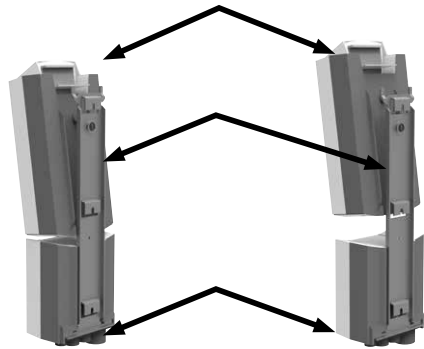
Caution!

Do not use the lift for at least one hour before replacing the battery.

Never expose the battery to extreme temperatures. Never leave the battery charger in damp/humid areas.



Installing the battery:

- Align the bottom of the battery with the top of the controller.
- Push the top of the battery forward until you hear a click. The battery is now in place.
- Gently push the top of the battery forward to make sure it is securely attached to the holder.

Removing the battery

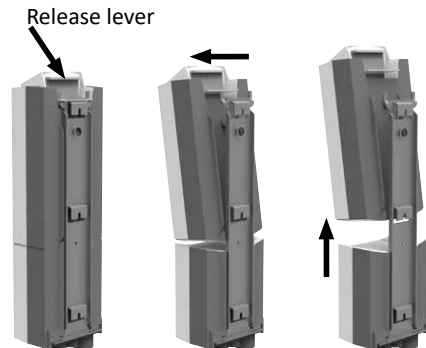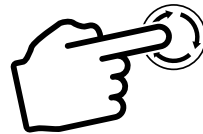- Press the emergency stop button.
- Hold the battery by the handle and press the release lever
- Remove the battery by lifting it off the holder

Diana II Comfort

Battery charge display and charging

Caution!

- The product cannot be used while charging.
- During charging, the lifting function is locked.
- Do not use the lift for 1 hour after charging. This will prolong the life of the battery.
- The battery needs to be charged continuously for at least 24 hours in the following situations:
 - a) Before using the lift for the first time.
 - b) Before being placed in storage (up to 3 months) without a power supply connection.
 - c) Immediately after a storage period.

Battery charge indicator

The battery charge indicator has three status indications. The battery indicators are either yellow or green until switched off (2 minutes after use).

Battery status 1:

The battery is full – no recharging required (100-50%): the third indicator is green.

Battery status 2:

The battery is okay (50-25%) – recharging required: the second indicator is yellow.

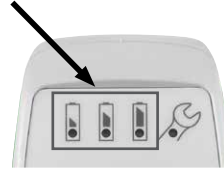
Battery status 3:

The battery is low (less than 25%) – recharging required: the first indicator is yellow and a buzzer sounds whenever a button is pressed.





Battery status 4:

The battery requires recharging. Some functions of the lift are lost and it is only possible to lower the lift arm. When in continued use, the indicator flashes and an audio signal is emitted.

The battery charge is indicated on the hand control.

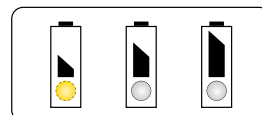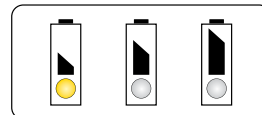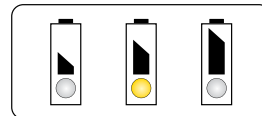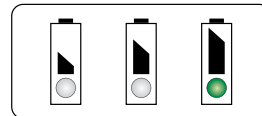
Battery level indicators on the hand control

Charging the battery

If you need to charge the battery, you need to:

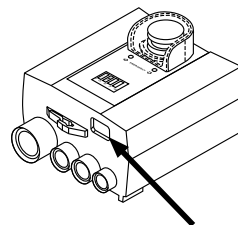- Press the emergency stop button
- Connect the black charging cable that is supplied to the control box
- Then insert the plug into a wall socket that is located in a dry room.


Connection for charging cable

When the rechargeable battery is connected, the LED display on the control box will come on.

The green LED indicates that the battery charger is connected to the mains. The yellow LED indicates the charging process. It takes approximately 5 hours to fully charge the battery.

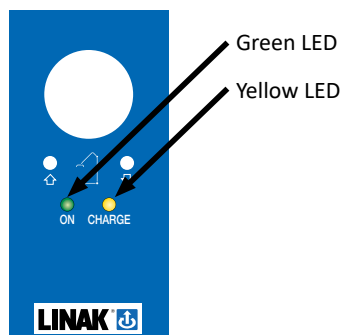Location of LED on control box


Charging the battery with an external charger (optional).

You can always purchase an external charger, for example to have a spare battery to hand. To charge the battery, it needs to be inserted in the external charging station. Ensure that the battery is properly inserted into the charging station. A light will come on if it is attached correctly.

If the battery emits a short beep, it needs to be charged. If a green light turns on when the battery is inserted, the battery is full. If the orange light turns on, the battery needs to be charged. Charging will stop automatically when the battery is full. This means you cannot overcharge the battery. It is best to charge the battery every night.

The external charger can be easily mounted by means of the wall mounting plate. It can be mounted on the wall near a 230 V socket.

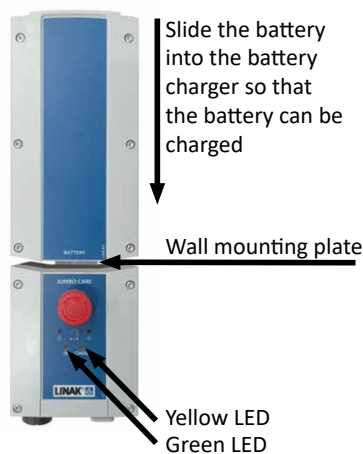We always recommend you purchase a spare battery/charger, so that you will always have access to a full battery. If one battery is empty, you can continue to work using the second battery.

Charging the battery must not take place in a wet/humid environment (shower area).

Diana II Comfort

Leg adjustment

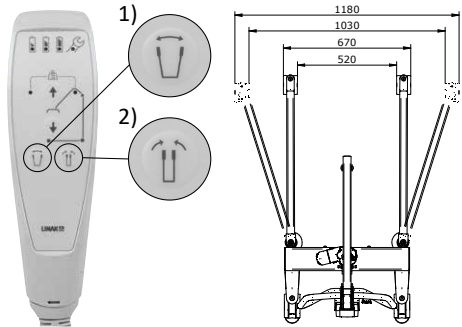
The leg adjustment function is useful when you need to move the lift to a client on a chair or other piece of furniture. The width of the chassis can be adjusted, so that the legs of the lift can fit around furniture. This means that the lift arm and hanger bar are closer to the client, making it easier to attach the sling and therefore ensuring a more comfortable lifting process.

Caution!

Keep sufficient distance from people and objects when using the leg adjustment function.

Press the buttons on the hand control to open (1) or close (2) the legs of the lift.


Castor brakes

The brakes are important components of the lift. The brakes are mounted on the rear two castors and prevent unwanted movement during use of the lift to ensure a stable starting position.

The brake is applied by using your foot to push down the red lever.

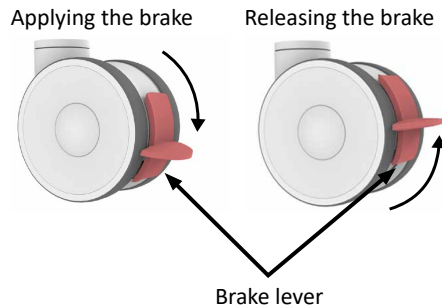The brake is released by using your foot to push up the red lever.


Tilting the spreader bar

The spreader bar can be tilted, which is useful when, for instance, a client is being lifted from a recumbent position. It means that the sling can be attached when the client is lying down and this position doesn't need to be changed until required. For example, the client can be placed in a sitting position when the destination of the transfer is a chair or toilet.


Electric 4-point spreader bar

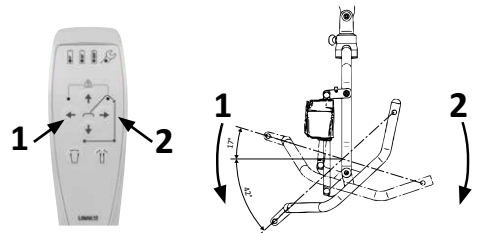The 4-point spreader bar can be used to change the client's position at any time during the lifting process by pressing the appropriate buttons on the hand control. The spreader bar remains in the same position when the button is released.


The spreader bar has a rotation lock (1) in 4 positions (each 90°).

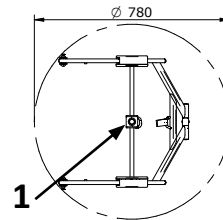The maximum turning radius of the spreader bar is approximately 790 mm.

Mechanical 4-point spreader bar (4MSB)

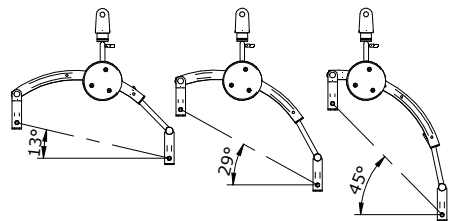Using the mechanical 4-point spreader bar, it is possible to position the client in three positions by using the adjustment strap.

Diana II Comfort

Use

CAUTION!

The assembled lift arm may only be raised by the lift actuator. Manual lifting the lift arm could damage the lift actuator.

Do not use the lift in an environment where other radio frequency emitting devices are used. Like any other electronic device, the lift can use and emit radio frequency energy. If the lift is not used in accordance with the instructions in the user manual, it may cause interference with nearby devices. The manufacturer cannot guarantee that interference will not occur in a particular location. In order to check if the lift is the source of any interference, change its location or disconnect the battery. If you still notice or suspect a fault, please try the following:

- Move the lift to another location;
- Increase the distance between devices;
- Contact the service-desk.

Moving the Diana II Comfort

The push handle must be used for moving and maneuvering the Diana II.

Think about your posture:

- Walk behind the lift when pushing straight ahead.
- Start moving the lift by using your own body weight as much as possible.
- Keep your back as straight as possible.
- When you need to make a turn, keep one hand on the push handle and use your foot to gently push the leg of the base in the correct direction.

Caution!

Always fit the sling as indicated in the user manual for the sling.

Push handle

Lifting the client

1. Position the lift with the correct hanger bar close to the client.
 2. Check whether the client is in a safe position.
 3. Lower the lift arm and hanger bar using the hand control to a position where it does not touch the client but it is possible to attach the loops or safety clips to the hanger bar without having to use physical force.
 4. Attach the loops or safety clips of the sling to the hanger bar. Attach the loops or clips for the shoulders first, followed by the legs.
 5. Carefully check each of the attachments. Using the hand control, raise the lift arm/hanger bar until there is some tension in the sling but do not start lifting the client yet. Check the attachments again.
-  6. Ensure the brakes are not engaged when lifting the client. The lift should be able to move to its centre of gravity.
- Note:** Brakes should be locked when lifting a client from the floor.
7. Lift the client by pressing the up button on the hand control. When using the 4-point spreader bar, the client can be moved into the preferred position: a recumbent, semi-recumbent and sitting position.

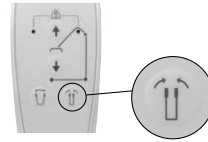
Diana II Comfort

Transferring the client

1. Move the lift backwards so that the legs can be closed again (2nd button on the hand control, figure 1).
2. Start moving the client. For the comfort of the client it is better to keep to short distances.
3. Once you have reached your destination, you can open the legs of the lift (if necessary) before getting close to another object (e.g. chair/toilet).
4. Using the electric 4-point spreader bar, place the client in a sitting position when he/she is above the chair or wheelchair or into a recumbent position if the client is above his/her bed.
5. Lower the client by holding down the relevant down button until the client is in a safe position.


1)

Detaching the sling

1. Only detach the sling when the client is fully placed on a solid surface such as a bed or wheelchair.
2. Ensure that the spreader bar cannot hit the client and then detach the sling from the hanger bar.
3. Check that all connection points are loose before doing anything else.
4. Move the lift away and park it in a safe place.
5. When the lift is parked, the brakes of the castors can be applied.

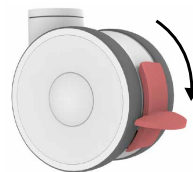2)

Maintenance instructions

Storage and transport

The lift must be kept in a cool and dry room. The climate conditions of the room must be within the following parameters:

- Temperature 10-40 °C (recommended 20 °C or lower);
- Humidity 30 ÷ 75%;
- Air pressure: 700 ÷ 1060 hPa.

Do not use the lift in environments where flammable materials may be present, e.g. in storage areas for flammable substances.

Cleaning

Cleaning is an important requirement to ensure that the lift works properly over a long period of time. Routine cleaning of the lift will suffice if it is used by the same client.

The lift will need to be disinfected if there are visible signs of material or possibly infected material (blood, stool, pus) or in situations requiring a high level of high infection control.

CAUTION!

The Diana II Comfort must be cleaned and disinfected every time a different client uses the lift before it can be used again.

CAUTION!

Before cleaning the lift, check that:

- All plugs are correctly connected.
- The battery is mounted on top of the control box.
- None of the electrical components show external damage. Failure to do this may result in water or cleaning agents to get into the lift and cause interference with the unit or damage to electrical components.
- Electrical components cannot be washed with water jets or high-pressure cleaners, etc. They can only be cleaned with a damp cloth.
- Stop using the lift if you suspect that water or liquid has entered the electrical components of the lift and report the incident immediately to the service center.
- Failure to comply with the above rules may result in serious damage to the equipment and could have other unforeseen consequences.

Diana II Comfort

Cleaning contact surfaces

1. The surface of the handles must be cleaned and maintained as follows:
 - a) undo the straps and remove all parts that do not belong to the lift.
 - b) clean surfaces with mild and environmentally-friendly cleaning agents
 - c) the same applies to cleaning the hand control.
2. Accessible areas of the push handles and the frame can be disinfected using mild and environmentally-friendly detergents
3. The castors only need to be disinfected if there are visible traces of infected or potentially contaminated material.

Do not use:

- pastes, waxes, sprays,
- strong detergents, solvents and detergents containing solvents, alcohol and leather cleaners.

Using such agents can also cause hardness and cracks in the material.

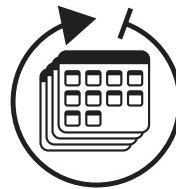
Any changes to the surface sheen are not covered under the warranty!

Servicing

An annual service and maintenance by authorized personnel is required every 12 months.


Human Care cannot be held responsible for damage caused by willful damage, overdue maintenance and incorrect use, or any resulting consequential damage.

We recommend checking the lift regularly for loose parts. Regularly clean the castors.

Maintenance

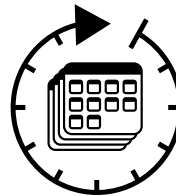During long periods of inactivity, we recommend checking all electrical and mechanical parts once a month by carrying out a lift test without the client. Additionally, the charging cables should be visually checked for possible damage after each mechanical lift or after moving the lift to a different location.

Maintenance of the frame

The metal sections of the frame can be cleaned with a soft, damp cloth. Cleaned surfaces should be wiped dry each time. Do not use cleaning products containing alcohol.

All moving joints must be lubricated once every six months or when they start making a loud noise when used. The moving joints include:

- castors and actuators;
- bearing bushes and joints of the legs and lift arm.

As a lubricant, we recommend using commercially available penetrating and lubricating products (e.g. Wurth HHS 2000). Any spillage of such products must be removed immediately with a dry cloth.

Threaded connections should be inspected periodically – once every six months – and, if necessary, any in-advertent tolerances must be remedied. Any in-advertent tolerances must be reported to the manufacturer's service center. The lift can no longer be used until the cause of this tolerances has been removed. Any tolerance in joints that cannot be remedied must be reported to the manufacturer's service department. The product can no longer be used until the cause of this slack has been removed.

Periodic electrical safety tests

A service is required every 12 months or 8000 lifting cycles, whichever comes first and every time after a fault/repair of the lift. Services, including the electrical safety test, must be carried out by a certified person.

Services should include as a minimum:

Check whether there is mechanical damage to the wiring;

Check whether there is mechanical damage to the housing of the controller, battery and actuator;

Check whether the height adjustment and leg adjustment buttons are working properly.

Troubleshooting

Problem	Solution
The lift does not respond to the hand control function	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the hand control cable is connected
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the other wires are connected
	Check whether the service LED is flashing
	Check whether the lift responds to the raising/lowering button on the controller
	Contact the service department
The lift does not respond to the control box buttons	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the other wires are connected
	Check whether the service LED is flashing
	Contact the service department
The lift is not charging	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the service LED is flashing
	Contact the service department
The lift stops while lifting the client	Check whether the overload indicator is flashing
	Check whether the battery is sufficiently charged
	Check whether the service LED is flashing
	Check whether the lift responds to the raising/lowering button on the controller
	Manually lower the lift
	Contact the service department
The lift is making strange noises (cracking, crossover, etc.).	Contact the service department
The lift cannot be moved	Check whether the brakes on the rear castors are locked
	Contact the service department

If the error messages persist, stop using the lift immediately and contact the distributor for further instructions.

Recycling

The Diana II Comfort is an electronic product covered by Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). The product label shows the WEEE symbol, which consists of a crossed-out wheeled bin. This label indicates that the batteries must be collected separately at the end of the lift's lifecycle. This requirement applies to the European Union. Do not dispose of these products in household waste.

Caution!

Batteries and electronics contain lead and must be recycled according to WEEE 2002/96/EC.

Metals

Electrical and electronic waste (WEEE)

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