**HUMAN CARE**  
Your Life. Your Way.

## Diana II MLA 90555

Service manual / Servicemanual / Servicehandbuch /  
Servicehandleiding

## Content

ENGLISH

Service Manual	3
Compliances and Standards	4
Useful information	4
Service manual description	5
Tools & Maintenance	5
Troubleshooting & Solving	7
Spare parts list	10

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# Service Manual

## Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

***Product modification is not permitted. Before use, always check that the product is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

## Compliances and Standards

---

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 10535.

## Useful information

---

### a) Service manual

The service manual is created for technicians involved in servicing of Diana II MLA mobile lift. Safe performance of mobile lift service procedures requires experience, technical skills and general knowledge of safety rules.

This manual uses warnings and cautions to pay special attention to service procedures.

**Warning** – this information tells us about the possibility of safety risk of technician and user.

**Caution** – this information tells us about actions that can cause potential damage of the lift.

### b) Diana II MLA installation and operating

For installation and operation of the lift, see the user manual.

### c) Service of Diana II MLA mobile lift

The service can be only made by Human Care's authorized service team or authorized units by Human Care.

Human Care shall bear no responsibility for the consequences of repairs or modifications that have been done by unauthorized personnel and for the use of non-original replacement parts dedicated by Human Care for mobile lift.

### d) Service documentation

Service documentation and service repair document are available on a website accessible for authorized service.

### e) Safety precautions

Mobile lift design provides maximum of safety. Safety of the device are ensured by safety standards IEC60601-1.

Read these cautions carefully before start servicing!

**Note** - do not perform service repairs and functional testing when the client is on the device.

### f) Additional information

In case of any problems with mobile lift contact the intermediary or manufacturer's service.

#### Service contact details:

HUMAN CARE SWEDEN (HQ)

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117 43 Stockholm

Phone: +46 8 665 35 00

Fax: +46 8 665 35 10

Email: [info.se@humancaregroup.com](mailto:info.se@humancaregroup.com)

## Service manual description

Service manual contains information about repairing, maintaining and identification of Patient Lift. The lift can be identified by using

the identification code which is placed on the name plate.

## Tools & Maintenance

**i** **Note** – if the plug from battery charger does not match a socket in your country, you can use an adapter.

### 3.1 Tools

Standard set of tools is required for servicing of mobile lift:

Allen key set

Flat screwdriver

Hammer

Ordinary set of various keys

Volt meter

# Diana II MLA Service Manual

## Device maintaining

All movable joints must be lubricated every six months or when experiencing squeaks or other noise. These nodes are:

- axle transport wheels,
- axle actuator mounting,
- jib arm bearing sleeve.

We recommend using any available penetrating lubricants.

Report all loose connections that are supposed to be fixed to the manufacturer's authorized service agent and stop using the product until the problem is resolved.

This device is equipped with electric actuator which allows to adjust the height of the jib arm. In order to ensure proper performance of the actuators check the actuators for unusual sounds every 3 months. These unusual sounds contain rattles, squeaks, creaks. Any perceived performance deficiency must be reported to Human Care technical service immediately ceasing device operation until remove their causes.

Not less frequently than every 1 years and after each failure/repair of actuator technical service must check the lift for electric safety.

The minimal range of the review should include:

- check if there were no mechanical wiring damages of the device,
- check if there were no mechanical damages of the controller and the actuators.

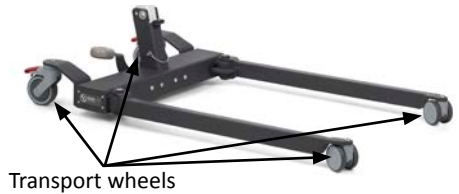
See checklist in user manual.

## Basic periodic review

One year after production date or every 8000 cycles of using the lift, the indicator lamp light up.

If this LED is on, operator must check condition of the lift (cables connections, looseness on the frame etc.).

Press and hold "up" and "down" button for 5 sec., to turn off the light after review.

# Troubleshooting & Solving

## Troubleshooting

Symptom	Possible cause - solution
Device doesn't work properly.	Diagnose the mobile lift.
Height adjustment doesn't work.	Replace main actuator.
Spreading legs emits sounds.	Lubrication legs spread mechanism.
Problem with moving device.	Check/replace castors.

### Diagnose the mobile lift

In case of problems with height adjustment please complete the following steps:

#### Step 1:

- Check the battery level on the hand control. If any led is lighting, please check the battery with the voltmeter,
- If the battery is empty, please charge it,
- If after a few hours the battery is still empty, please check if the charger is working correctly by using the voltmeter. If the charger is ok, you have to replace the battery for a new one.

Checking voltage on battery

#### Step 2:

If the battery level is correct, check that the wires are connected properly to the control box in the bottom part of the device.

Checking voltage on charger

#### Step 3:

If everything looks good so far but device still doesn't work, for further diagnose, spare parts will be necessary (hand control, control box and actuator).

- At first, replace the hand control for new one (or switch with another device if it's possible) and check.
- Next, prepare a new control box and connect it with slots from device,
- Now test what is wrong, control box, hand control or actuator.

Checking wires connection

# Diana II MLA Service Manual

## Step 4:

To remove the control box, just loosen two main screws and lift up whole holder with the control box.

Loosening two screws

Removing main holder

## Replace main actuator

### Step 1:

Remove the cotter from upper side and take out the safety pin. Do the same operation for bottom side of actuator.

1. Removing cotter

2. Removing safety pin

3. View after removal

4. Removing cotter

5. Removing safety pin

6. View after removal

**Step 2:**

Remove old and install the new actuator in the reverse order to dismantling, carefully tighten the gland.

**Lubrication of bottom leg mechanism**

**Caution** - Following procedures require turn down device on aside. To protect it from any damage or scratches we strongly recommend to prepare some kind of protection (card boards, sponges, etc.) on the floor.

Sponges under device

**Step 1:**

Remove the bottom metal cover. Use the pedals for localize noises.

Removing the screws

Internal view

**Step 2:**

Lubricate with oil, or another lubricant moving part.

Lubricating moving parts

# Diana II MLA Service Manual

## Wheel replacement

To replace the rear wheels, use the allen and 17" key. To replace front wheels use 17" socked wrench.

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