HUMAN CARE

Your Life. Your Way.

Diana II Comfort 4ESB 90560

Service manual / Servicemanual / Servicehandbuch /
Servicehandleiding

Content

ENGLISH

Maintenance Manual	3
Compliances and Standards	4
Useful information	4
Service manual description	5
Tools & Maintenance	5
Troubleshooting & Solving	7
Spare parts list	15

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Maintenance Manual

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CE MARKING symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

RECYCLE according to local regulations.

READ INSTRUCTIONS and ensure you fully understand them before using this product.

***Product modification is not permitted.
Before use, always check that the product
is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at www.humancaregroup.com.

Contact your local distributor if you have any questions about the product and its use.

Diana II Comfort 4ESB Service Manual

Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This sling has been tested in accordance with ISO 10535.

Useful information

a) Service manual

The service manual is created for technicians involved in servicing of Diana II Comfort 4ESB mobile lift. Safe performance of mobile lift service procedures requires experience, technical skills and general knowledge of safety rules.

This manual uses warnings and cautions to pay special attention to service procedures.

Warning – this information tells us about the possibility of safety risk of technicians and user.

Caution – this information tells us about actions that can cause potential damage of the lift.

b) Diana II Comfort 4ESB installation and operating

For installation and operation of the lift, see the user manual.

c) Service of Diana II Comfort 4ESB mobile lift

The service can be only made by Human Care's authorized service team or authorized units by Human Care.

Human Care shall bear no responsibility for the consequences of repairs or modifications that have been done by unauthorized personnel and for the use of non-original replacement parts dedicated by Human Care for mobile lifts.

d) Service documentation

Service documentation and service repair document are available on a website accessible for authorized service.

e) Safety precautions

The mobile lift design provides maximum of safety. Safety of the device are ensured by safety standards IEC60601-1.

Read these cautions carefully before start servicing!

Note - do not perform service repairs and functional testing when the client is on the device.

f) Additional information

In case of any problems with the lift, contact the intermediary or manufacturer's service.

Service contact details:

HUMAN CARE SWEDEN (HQ)

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117 43 Stockholm

Phone: +46 8 665 35 00

Fax: +46 8 665 35 10

Email: info.se@humancaregroup.com

Service manual description

The service manual contains information about repairing, maintaining and identification of the mobile lift. The lift can be identified by using

the identification code which is placed on the name plate.

Tools & Maintenance

Note – if the plug from battery charger does not match a socket in your country, you can use an adapter.

Tools

Standard set of tools is required for servicing of the mobile lift:

Allen key set

Flat screwdriver

Hammer

Ordinary set of various keys

Volt meter

Diana II Comfort 4ESB Service Manual

Device maintaining

All movable joints must be lubricated every six months or when experiencing squeaks or other noise. These nodes are:

- axle transport wheels,
- axle actuator mounting,
- jib arm bearing sleeve.

We recommend using any available penetrating lubricants.

Report all loose connections that are supposed to be fixed to the manufacturer's authorized service agent and stop using the product until the problem is resolved.

This device is equipped with electric actuator which allows to adjust the height of the jib arm. In order to ensure proper performance of the actuators check the actuators for unusual sounds every 3 months. These unusual sounds contain rattles, squeaks, creaks. Any perceived performance deficiency must be reported to Human Care technical service immediately ceasing device operation until remove their causes.

Not less frequently than every 1 years and after each failure/repair of actuator technical service must check the lift for electric safety.

The minimal range of the review should include:

- check if there were no mechanical wiring damages of the device,
- check if there were no mechanical damages of the control box and the actuators.


See checklist in user manual.

Basic periodic review

One year after production date or every 8000 cycles of using the lift, the indicator lamp light up.

If this LED is on, operator must check condition of the lift (cables connections, looseness on the frame etc.).

Press and hold "up" and "down" button for 5 sec., to turn off the light after review.

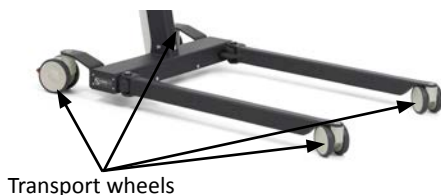Troubleshooting & Solving

Troubleshooting

Symptom	Possible cause - solution
Device doesn't work properly.	Diagnose the mobile lift.
Height adjustment doesn't work.	Replace main actuator.
The base legs doesn't spread.	Replace bottom actuator.
Holder is not working.	Replace holder actuator.
Problem with moving device.	Check/replace castors.

Diagnose the mobile lift

In case of problems with height adjustment, please complete following steps:

Step 1:

- Check the battery level on the hand control. If any led is lighting, please check the battery with a voltmeter,
- If the battery is empty, please charge it,
- If after a few hours the battery is still empty, please check if the charger is working correctly by using the voltmeter. If the charger is ok, you have to replace the battery for a new one.

Checking voltage on battery

Checking voltage on charger

Step 2:

If the battery level is correct, check that the wires are connected properly to the control box in the bottom part of the device.

Checking wires connection

Diana II Comfort 4ESB Service Manual

Step 3:

If everything looks good for so far, but the device still doesn't work, for further diagnose, spare parts will be necessary (hand control, control box and actuator).

- At first, replace the hand control for a new one (or switch with another device if possible) and check,
- Next, prepare a control box and connect it with slots from the device,
- Now test what is wrong: control box, hand control or actuator.

Note – in case of the situation that one of actuators is not working, make sure that problem is with the actuator, by swapping the wires 1 and 2 for actuators in the control box and use the remote control to test (a working actuator means a faulty controller).

Step 4:

To remove the control box, just loosen two main screws and lift up whole holder with control box.

Loosening two screws

Removing main holder

Replace main actuator

Caution - Following procedures require turn down device on aside. To protect it from any damage or scratches we strongly recommend to prepare some kind of protection (card boards, sponges, etc.) on the floor.

Sponges under device

Step 1:

Take out blockage from plug with small screwdriver and take out the plug.

View of a plug

Removing the blockage

Unplugged

Diana II Comfort 4ESB Service Manual

Step 2:

Use a flat screwdriver to remove the star lock cap and circlips from the upper rod holding the actuator (from only one side of the pin) and remove the pin. Do the same operation for the second connection actuator of the frame.

Removing star lock cap

Removing circlip

Removing pin

Removing pin (second side)

Removing circlip (second side)

Removing pin (second side)

Step 3:

Remove old and install the new actuator in the reverse order to dismounting, carefully tighten the gland.

Replace bottom actuator

Caution - Following procedures require turn down device on aside. To protect it from any damage or scratches we strongly recommend to prepare some kind of protection (card boards, sponges, etc.) on the floor.

Step 1:

Remove bottom metal cover. With hand control try to spread legs and check with actuator does not work.

Sponges under device

Removing the screws

Removing the screws

Step 2:

With hand control try to spread legs and check with actuator does not work.

Internal view

Diana II Comfort 4ESB Service Manual

Step 3:

Remove only two screws from failure actuator's side (two top screws in that occasion), pull up the actuator and remove the bottom pins holding the actuator.

Removing first screw

Removing second screw

Removing safety ring

View after disconnecting actuator

Step 4:

Disconnect the cable

Take out actuator from the frame

Use flat screw driver


Take out blockade


Removing plug

Step 5:

Remove second safety ring and pin.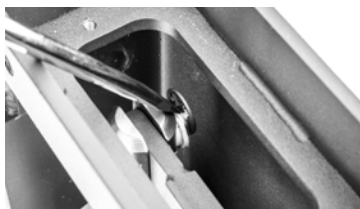

Removing first screw

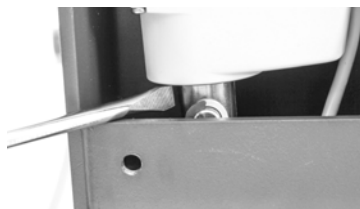Removing second screw

Step 6:

Remove old and install the new actuator in the reverse order to dismantling, carefully tighten the gland.

Replace of the holder actuator**Step 1:**

Use a flat screwdriver to unlock the plug and disconnect wire with actuator.

Removing blockade step 1

Removing blockade step 2

Removing blockade step 3

Disconnecting the cable

Diana II Comfort 4ESB Service Manual

Step 2:

Use a flat screwdriver to remove the star lock cap and circlips from the upper rod holding the actuator (from only one side of the pin) and remove the pin. Do the same operation for bottom side of actuator.

Removing star lock cap

Removing circlip

Removing pin

Removing star lock cap(second side)

Removing circlip (second side)

Removing pin(second side)

Wheel replacement

To replace rear wheels, use a socked wrench size 19". To replace front wheels, use an allen key and socked wrench size 17".

Rear wheels replacement

Front wheels replacement

Spare parts list

INDEX	Price list for Diana II Comfort 4ESB
04-N0101-0012	LINAK Battery BAJ
04-N0107-0020	LINAK 3 functions hand control HB3X
04-N0109-0040	LINAK CBJ-CARE control box
04-N0301-0071	LINAK hanger bar actuator LA23
04-N0301-0072	LINAK main actuator LA34
04-N0301-0080	LINAK secondary actuator LA23
04-N0901-0035	LINAK micro-fit grey wire 750mm
04-N0901-0018	LINAK external charger
04-N0901-0043	LINAK control box holder + screw M5x8
04-N0901-0055	LINAK control box splitter
04-N0901-0094	LINAK safety ring for micro-fit wire
04-N0901-0096	LINAK safety lock for secondary actuator's plug
04-N0901-0114	LINAK mini-Fit gray wire 500mm
04-N0901-0123	EU LINAK power supply cord
04-M2010-0064	TENTE double wheel set D125 White with red brake
04-M2010-0066	TENTE double wheel set D100 white
04-M3010-0013	Spring plate DIN 6799- 8
04-M3005-0030	Outer snap ring steel 10 with a hook
04-E4043-0047	Grommet rubber D10x7
04-E4041-0071	Gray gland with metric thread M12x1.5
04-M3010-0014	Retaining plate (pin on shaft) 10 steel
04-E4035-0016	The bridge of the stretch
04-E4037-0009	Cable clamp
04-E4041-0050	Black gland with metric thread M12x1.5
04-E4043-0045	Rubber grommet 18.5 mm
02-OBR-1203	Hook sling
04-M2501-0037	O-ring seal 35x3
04-M3023-0016	Indexing plunger with lock D8x8 black

Diana II Comfort 4ESB Servicemanual

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