



Optimal support
Safe
Two materials



High Basic Sling

Our High Basic Sling is a comfortable sling and a great solution for clients who have low upper body stability. The design of the High Basic Sling is similar to the Basic Sling with the exception that it provides support to the head and neck during transfers.



Lots of support

The High Basic Sling is perfect for those clients who need a lot of support, and who have low or high tone. The sling makes transfer situations safe for both carer and client and is well suitable for lifting clients to and from the floor. In order to provide optimal support for the head and neck, the sling is equipped with two removable slats.

transfer is complete. The Polyester Net fabric is flexible, airy and dries quickly which makes it suitable for use in bathing environments. They can be used with a 2-point, 3-point or 4-point suspension depending on the need.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.



Two fabrics

The High Basic Sling is a comfortable sling that is available in two fabric choices - Polyester and Polyester Net. The Polyester Net materials allow moisture and heat to transfer efficiently from the client which allows the sling to be left behind the client for a short period of time after the



Walking Aids



Healthcare Beds



Lifting Solutions



Hygiene



Convertible Chairs

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.: 21180-*



Leg Padding and Supports

Ideal for users with sensitive skin or pain. Part no.: 21146-S, M, L Reinforced supports. One size. Part no. 25102



Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866



Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 (2- or 3-Point - Quick Release), 55619 (4-Point - Attach to leg hook)

Lifting Situation




Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.




Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.

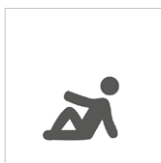

Use to/from the floor

Suitable for transfers from the floor.

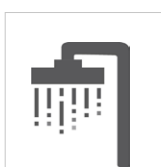

Use for showering

Only Polyester Net fabric can be used for this application.

Care and Features




Polyester Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering..



Polyester

Durable material with a low coefficient of friction, making it easy to apply and clean. Equipped with handles.

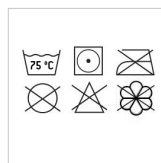


Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering

Technical specifications

Article number	Size	Colour	A	B	C	
Polyester net	Polyester					
25037-JS	25032-JS	Junior Small	JS	22 cm / 8,7"	62 cm / 24,4"	67 cm / 26,4"
25037-JM	25032-JM	Junior Medium	JM	28 cm / 11"	65 cm / 25,6"	74 cm / 29,1"
25037-S	25032-S	Small	S	35 cm / 13,8"	77 cm / 30,3"	86 cm / 33,8"
25037-M	25032-M	Medium	M	40 cm / 15,7"	88 cm / 34,6"	91 cm / 35,8"
25037-L	25032-L	Large	L	44 cm / 17,3"	98 cm / 38,6"	100 cm / 39,4"
25037-XL	25032-XL	Extra Large	XL	48 cm / 18,9"	102 cm / 40,1"	102 cm / 40,1"
25037-XXL	25032-XXL	Extra Extra Large	XXL	52 cm / 20,5"	108 cm / 42,5"	107 cm / 42,1"
Maximum user weight: 300kg/660lbs						

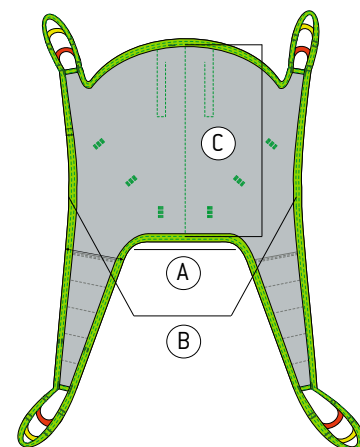

* Available in assorted lengths: 10cm/3.9" (21180-10), 15cm/5.9" (21180-15), 20cm/7.8" (21180-20), 25cm/9.8" (21180-25), 35cm/13.7" (21180-35)



*We improve freedom of movement
and quality of care for people.*

www.humancaregroup.com



HUMAN CARE SWEDEN (HO)

Årstaängsvägen 21B
117 43 Stockholm
+46 8 665 35 00
info@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
ON K2E 7K1 Ottawa
+1 613 723 6734
info.ca@humancaregroup.com

HUMAN CARE THE NETHERLANDS

Elspeterweg 124
8076 PA Vierhouten
+31 577 41 21 71
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron rd, Suite K
TX 78754 Austin
+1 512 76 7199
info.us@humancaregroup.com